 **ceramill® XTP**

 **ceramill® XTP**
Model

 **ceramill® XTP**
Ortho Model

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EN Instruction Manual 21 - 38

FR Instructions d'utilisation 39 - 56

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AMANNGIRRBACH

- Original Gebrauchsanweisung -

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- Translation of the original Operating Instructions -

Table of Contents

Symbol explanation	22
General safety instructions	25
Characteristics	27
Warranty/Disclaimer	30
Testing the printing resins	30
Application	31
Process steps in the dental laboratory	32
Environmental protection	38
Download information	38

EN

Symbol explanation

Warnings

Warnings in the text are marked with a warning triangle and are outlined.

In the event of danger due to electricity, the exclamation mark in the warning triangle is replaced by a lightning symbol.

Signal words at the beginning of a warning indicate the type and severity of the consequences if the measures to avert the danger are not followed.

- _ **NOTE** means that property damage can occur.
- _ **CAUTION** means that minor to moderate personal injury may occur.
- _ **WARNING** means that serious personal injury can occur.
- _ **DANGER** means that life-threatening personal injury can occur.

Important information

Important information that does not pose a risk to people or property is marked with the symbol below. They are also surrounded by lines.

Further symbols in the instructions

Symbol	Meaning
▷	Item of an action description
–	Item of a list
▪	Sub-item of an action description or a list
[3]	Numbers in square brackets refer to location numbers in graphics

Other symbols on the product

Symbol	Meaning
	Item number
	Batch code
	Manufacturer
	Country of manufacture
	Observe instruction manual
	Usable until
UFI	Unique Formula Identifier (UFI)
	Color
	Protect from sunlight
	Storage temperature
	Attention (color coding: red frame, white background, black symbol)

General safety instructions

Ceramill XTP Model (gold/pastel/grey)

WARNING:

Causes skin irritation.

Causes severe eye irritation.

CAUTION:

- ▷ Wear eye protection/face protection.
- ▷ **IN CASE OF CONTACT WITH THE EYES:** Rinse gently with water for a few minutes. If possible, remove any contact lenses. Continue rinsing.
- ▷ In the case of skin irritation: Seek medical advice/seek medical help.
- ▷ If eye irritation persists: Seek medical advice/seek medical help.
- ▷ Remove all contaminated clothing immediately and wash before wearing again.
- ▷ Dispose of contents/container in accordance with local/regional/national/international regulations.

Ceramill XTP Ortho Model**WARNING:**

Causes skin irritation.

May cause allergic skin reactions.

Causes severe eye irritation.

Harmful to aquatic organisms, with long-term effects.

**CAUTION:**

- ▷ Do not breathe dust/smoke/gas/mist/vapor/aerosol.
- ▷ Avoid release to the environment.
- ▷ Wear protective gloves/eye protection/face protection.
- ▷ IN CASE OF CONTACT WITH THE EYES: Rinse gently with water for a few minutes. If possible, remove any contact lenses. Continue rinsing.
- ▷ If skin irritation or rash occurs: Seek medical advice/seek medical help.
- ▷ Dispose of contents/container in accordance with local/regional/national/international regulations.
- ▷ Contains isocyanates. May cause allergic reactions.

Characteristics

Intended use

Light-hardening 3D printing resins for producing dental tooth models using stereolithography processes. The materials can be processed on UV-LED 385 nm/LED 405 nm (approved for the DLP printers Asiga Max UV and NextDent 5100).

Product description

Ceramill XTP Model (gold/grey/pastel) is a 3D printing resin and is suitable for producing 3D printed dental tooth models. The product is classified as a non-medical product. Ceramill XTP Ortho Model is a 3D printing resin and is suitable for the production of 3D printed models for thermoforming and orthodontic models. The product is classified as a non-medical product.

Safety data sheet/declaration of conformity

The safety data sheet of the products can be downloaded from the Amann Girrbach website under *Downloads > Product related Documents*.

Chemical composition

Ingredients	Weight %
(Meth-)Acrylate	> 95
Initiators, stabilizers, dyes, pigments, fillers	< 5

Shelf life

Ceramill XTP Model (gold/pastel/grey) and Ceramill XTP Ortho Model can be used for 2 years from the date of manufacture if stored appropriately.

The expiry date and batch number can be seen on the original packaging.

Storage

- ▷ Store the material in a cool, dark and well-ventilated place.
- ▷ Maintain storage temperature 2-28 °C.
- ▷ Do not use products after the expiry date.
- ▷ Always keep the container tightly closed and close carefully immediately after each use.
- ▷ Protect from heat and direct sunlight.
- ▷ To avoid premature hardening, only expose liquid material to daylight for a short time.
- ▷ When storing material containers that have already been filled, keep them closed and in the dark.
- ▷ Use material in material trays within two months of storage.

Warranty/Disclaimer

Application recommendations, whether given orally, in writing or in the course of practical instruction, are considered guidelines. Our products are subject to continuous further development and are tested according to the state of the art and legal requirements. We reserve the right to make any resulting changes in handling and composition.

The currently valid version of the instruction manual can be found under

www.amanngirrbach.com/instruction-manuals

This version replaces all previous versions.

Testing the printing resins

Upon receipt, the bottles of printing resins must be checked to ensure that they are in perfect visual condition. After using a (transport) damaged bottle or a (transport) damaged resin, a claim is no longer possible.

Application

Indication areas

Ceramill XTP Model (gold/grey/pastel) is suitable for the production of 3D printed dental tooth models such as stump models, implant models and control models.

Due to its high temperature resistance, Ceramill XTP Ortho Model is particularly suitable for the production of 3D printed models for the deep-drawing technology of splints/aligners, but also for orthodontic models.

Validated for the following DLP printers with corresponding polymerization devices

Device type	Model
DLP printer	_ NextDent 5100 405 nm
	_ Asiga Max UV 385 nm
Polymerization device	_ NextDent LC-3DPrint Box
	_ Otofash G171

NOTE:

For optimum results, the appropriate build style must be selected in the corresponding printer software (Ceramill XTP Model (gold/pastel/grey) or Ceramill XTP Ortho Model).

Contraindications

- _ Ceramill XTP Model/Ortho Model is intended for the fabrication of prosthetic/orthodontic models and not for other purposes.
- _ Not approved for use in the patient's mouth.

Deviations from these instructions for use may have a negative impact on the quality of the printed models.

Process steps in the dental laboratory

General information

Contaminated material trays can have a negative impact on the component geometry of the printed objects.

The properties of the end product depend, among other things, on the post-processing process. Correct post-exposure is important for optimum properties. Therefore, it must be ensured that the exposure device is in proper condition and that the objects are completely hardened.

Manufacturing process

Design and print settings

- ▷ After designing the models, select the appropriate build style for Ceramill XTP Model or Ceramill XTP Ortho Model in the software and transfer it to the printer.

Ceramill XTP Ortho Model: Video tutorials for blocking out the models for the deep-drawing technique can be found in the Amann Girrbach Academy at <https://academy.amanngirrbach.com/>

Before using the printing resins

- ▷ Before use, shake the bottle by hand for approximately 1 minute or mix for 10 minutes on the NextDent LC-3DMixer before using the printing resin.

Fill the material tray

The use of protective gloves is recommended before use.

- ▷ Fill the printing resin into the material tray of the corresponding 3D printer.

The recommended processing temperature is $23\text{ °C} \pm 2\text{ °C}$. Permanent exposure to all light sources should be prevented.

If the printing resin is stored in a closed material tray, the resin should be stirred again before each further use to ensure that it is free of streaks.

Different batches of the same product should not be mixed.

Remove printed objects from the build platform/pre-cleaning

- ▷ Remove the build platform from the device.
- ▷ Initially spray the components on the platform with ethanol or isopropanol.
- ▷ Remove the components from the platform using a suitable tool.
- ▷ Pre-clean the components ultrasonically in ethanol (> 98 %) or isopropanol for 3 minutes.

**CAUTION:**

Before using the ultrasound device, check whether it is suitable for flammable solvents.

Always pour ethanol or isopropanol into a beaker for safety reasons and do not pour it directly into the ultrasonic unit.

There is a risk of ignition if this is not observed.

Cleaning

- ▷ Dry the components with compressed air.
- ▷ Remove the supports.
- ▷ Clean the components for 2 minutes in fresh ethanol or isopropanol in an ultrasonic device.

CAUTION:

Before using the ultrasound device, check whether it is suitable for flammable solvents.

Always pour ethanol or isopropanol into a beaker for safety reasons and do not pour it directly into the ultrasonic unit.

There is a risk of ignition if this is not observed.

- ▷ Dry the components with compressed air once again.
- ▷ Allow the components to air dry for 10 minutes.

Post-curing

- ▷ For the required final mechanical properties, place the printed models in the appropriate polymerization devices for final polymerization.

Important note about the exposure times of the different polymerization devices:

Polymerization device	Exposure times
NextDent LC-3DPrint Box (NextDent 5100)	Light harden for 10 minutes
Otoflash G171 (Asiga Max UV)	2x 2000 flashes (after 2000 flashes: turn objects once)

Post processing

- ▷ Remove all support structures.
- ▷ If necessary, finalize the surfaces with conventional dental instruments.

Isolation recommendation for orthodontic models (Ceramill XTP Ortho Model)

- _ Vaseline
- _ Isolation agent SHERAiso-3d
- _ 3D model isolation Sheu

Environmental protection

Packaging

In the packaging, the manufacturer participates in country-specific recycling systems that ensure optimum recycling.

Disposal

- _ Ceramill XTP Model/Ortho Model must not be disposed of with household waste.
- _ Do not allow to enter drains.
- _ Dispose of contents/container in accordance with local/regional/national/international regulations.

Download information

Further instructions, safety data sheets and video tutorials can be found in the download area at www.amanngirrbach.com.

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Amann Girrbach AG
Amann Girrbach GmbH