
artex®/splitex®-eingipsgerät

Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Modo de empleo

| | |
|----------|---------|
| Deutsch | 02 - 03 |
| English | 04 - 05 |
| Français | 06 - 07 |
| Italiano | 08 - 09 |
| Español | 10 - 11 |

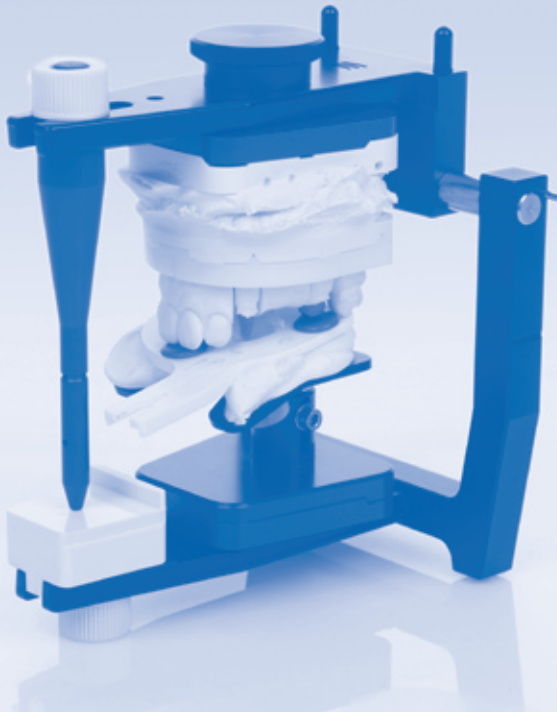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
1. MOUNTING THE MAXILLARY MODEL WITH TRANSFER TABLE


Put the transfer table with the secured facebow registration delivered from the dental surgery in the mounting articulator.

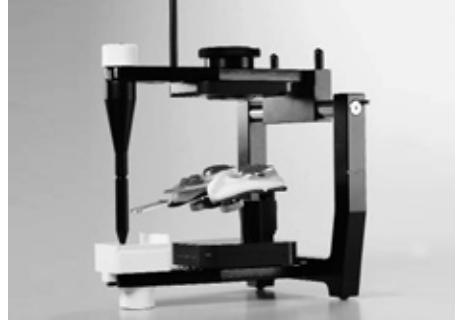
2.

Position maxillary model in the bite fork impressions. Insert Splitex counter plate in top part of mounting articulator. Check soace available between model and top part and regulate if necessary. (Trim down model if too little soace or mount in two stages if too much).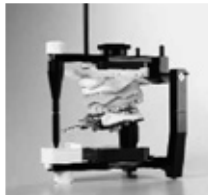



Moisten the base of the model. Mix an appropriate quantity of mounting plaster for the space to a creamy consistency. When mounting in two stages, apply the olaster to the model. Insert spacerfoil (bubble-wrap) between the counter plate and the mounting plaster. Close top part.

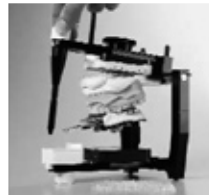Once the plaster has set, open top part and remove bubble wrap.


3.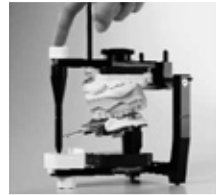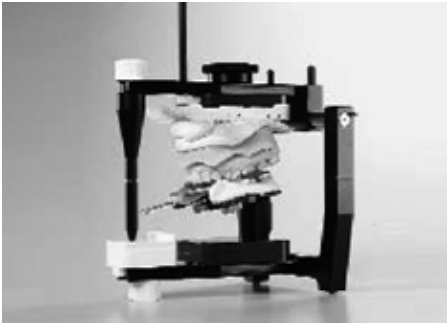

Mix plaster to a creamy consistency, moisten the set mounting plaster, apply plaster to counter plate and mounting plaster. Close top part.

Eliminate plaster resistance by knocking on the top part of the mounting articulator - relieve stress in plaster (do not press together!)

4.

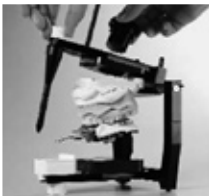Once plaster has set, pull out magnetic part with knurled handle with a twisting movement. Open mounting articulator. Remove maxilla from impressions. Remove transferable from the mounting articulator.

Mounting the mandibular model: Mounting articulator is upside down. The maxillary model is already mounted over the face-bow registration. Place the mandibular model via the teeth in centric occlusion and secure on the counter plate with plaster. Observe the same criteria as when inserting the maxillary model: distance from bottom part, quantity of plaster etc.

Notes

Models are always placed in the articulator with the centric closed or in the mounting articulator (axis of rotation). Incisal tables are generally used without clip attachments. Always mix mounting plaster to a creamy consistency so that the top part can be closed „without resistance“. Knock to relieve stress in plaster, never press or squeeze.

Ordering information:

| | |
|------------|---|
| Order No.: | |
| 216020 | Splitex Mounting Articulato 116 mm |
| 216020C | Splitex Mounting Articulato 126 mm for Artex Carbon |
| 216021 | O-Ring for head support |
| 216031C | Conversion kit for Mounting articulator C-Version |

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Amann Gırrbach AG
Amann Gırrbach GmbH