

# Cleanic<sup>®</sup>

The one step prophylaxis paste



## ALL IN ONE

Universal prophylaxis paste  
Guarantees outstanding  
Results



## PATENTED PERLITE TECHNOLOGY

Featuring integrated  
Abrasion variability



## SCIENTIFICALLY PROVEN

Effective cleaning & polishing with  
Low abrasion of Enamel and dentine



## TIME SAVING

From cleaning to polishing  
in one step



**Cleanic** is the one step prophylaxis paste that guarantees your patient outstanding cleaning & polishing results with low abrasion of enamel and dentine in a one-step procedure.

That's because of the clinically proven integrated abrasion variability of natural perlite particles



## WHERE TRADITIONAL PROPHY PASTES FALL SHORT...

Traditional pastes, usually based on pumice, are aggressive towards enamel, dentine and fillings, causing:

DISCOLOURATION  
AND FORMATION  
OF CALCULUS



SCRATCHES  
TO ENAMEL  
AND DENTINE



UNNECESSARY  
DAMAGE  
TO HARD TISSUE



PALPABLE LOSS  
OF TISSUE  
OVER TIME



They also require twice the work, needing at least two different grit pastes to be effective. This often results in polishing being omitted from the procedure in an effort to save time, with the remaining unpolished surface becoming discoloured over time and accumulating new plaque.






## WHAT'S DIFFERENT WITH CLEANIC?

Cleanic is a **one-step cleaning and polishing prophy paste** based on perlite technology.

This means gentle but efficient cleaning and polishing, all done in one session, with one paste.



Cleanic available in 5 tastes

-  Berry Burst
-  Mint
-  Green Apple
-  Light
-  Mint Fluoride Free

# INTEGRATED ABRASION VARIABILITY


The volcanic Perlite (natural silica) particles adapt their abrasiveness throughout your procedure.

## CLEANING

When Cleanic is first applied to the tooth surface, it behaves like a coarse paste. This is when Cleanic's cleaning ability is in full force.


## POLISHING

Under load and within seconds, the abrasive particles are fractionated and their cutting edges become rounded and less blunt. At this point, the paste becomes effective for a high gloss polishing.

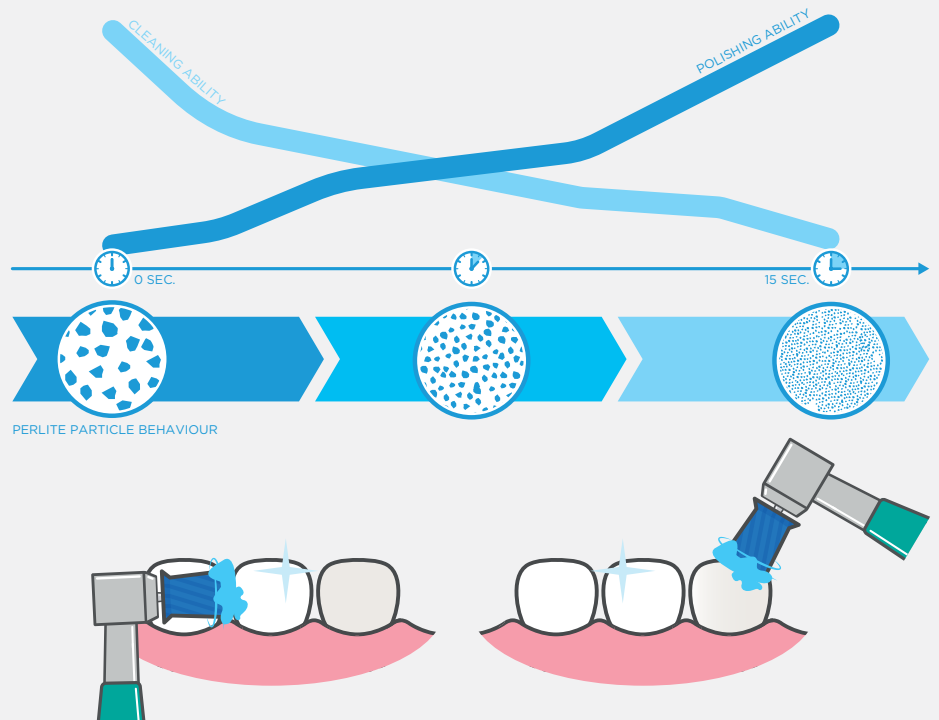

## HOW TO USE CLEANIC

Apply the Cleanic paste to a limited group of teeth at a time (about 3 teeth) using a rubber cup (i.e. Pro-Cups) to work it from cleaning to polishing, all in one step. Once done, apply a fresh portion of Cleanic paste to treat the next group of teeth.

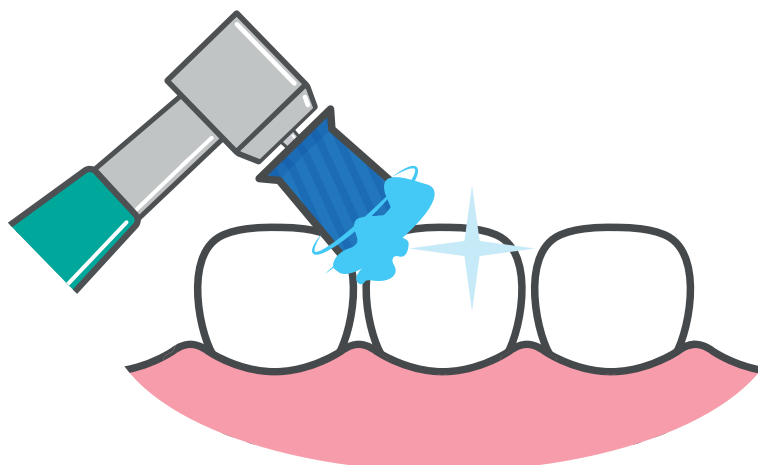

### FOR NORMAL DENTITION:

Work from the incisal/occlusal surface of each tooth, moving towards the gingival area; this will minimise abrasion on the tooth.

### FOR EXPOSED DENTINE OR ROOT SURFACES:

Start by applying a fresh portion of paste to the enamel surface. Move towards the exposed root surfaces or dentine with the used paste (so it is less aggressive).

## ORDER YOUR CLEANIC NOW!

| Part No. | Description                                    | Content  | Fluoride  | RDA | REA | Taste       | Menthol | Paste Colour |
|----------|--|--|-----------|-----|-----|-------------|---------|--------------|
| 3110     | Cleanic Cartridge Refill Packs                 | 200g   | 0.15% CaF | 27  | 3.4 | Mint        | Y       | Light blue   |
| 3130     | Cleanic Cartridge Refill Packs                 | 100g   | 0.15% CaF | 27  | 3.4 | Mint        | Y       | Light blue   |
| 3210     | Cleanic Cartridge Refill Packs - Fluoride free | 200g   | -         | 27  | 3.4 | Mint        | Y       | Light blue   |
| 3230     | Cleanic Cartridge Refill Packs - Fluoride free | 100g   | -         | 27  | 3.4 | Mint        | Y       | Light blue   |
| 3140     | Cleanic Single Dose                            | 200x 2g  | 0.10% NaF | 27  | 3.4 | Mint        | Y       | Dark blue    |
| 3180     | Cleanic in Tube Mint                           | 100g   | 0.10% NaF | 27  | 3.4 | Mint        | Y       | Blue         |
| 3182     | Cleanic in Tube Green Apple                    | 100g   | 0.10% NaF | 27  | 3.4 | Green Apple | N       | Green        |
| 3183     | Cleanic in Tube Mint Fluoride Free             | 100g   | -         | 27  | 3.4 | Mint        | Y       | Light blue   |
| 3184     | Cleanic in Tube Light                          | 100g   | 0.10% NaF | 27  | 3.4 | --          | N       | Light grey   |
| 3186     | Cleanic in Tube Berry Burst                    | 100g   | 0.10% NaF | 27  | 3.4 | Red fruits  | N       | Pink         |
| 3190     | Cleanic Collection Kit                         | 3x 100g Tube Cleanic Prophy Paste (Mint with Fluoride, Green Apple, Light), Tube Squeezer, Reusable Bag. |           |     |     |             |         |              |

## TRY CLEANIC WITH ITS IDEAL PARTNERS!

### Pro-Cup and Pro-Cup Junior

The laminated profile of these single-use cups helps avoid the formation of splash balls while applying the prophy paste. The same profile also helps mitigate overheating. Pro-Cup and Pro-Cup Junior are not made of natural latex.

|  | CODES & QUANTITIES |     |          |     | COLOUR | DESCRIPTION           | TYPE  |       | SINGLE USE |
|--|--------------------|-----|----------|-----|--------|-----------------------|-------|-------|------------|
|  | CODE               | QTY | CODE     | QTY |        |                       | LATCH | SCREW |            |
|  | 990/30             | 30  | 990/120  | 120 | ●      | Pro-Cup - Soft        | ✓     |       | ✓          |
|  | 991/30             | 30  | 991/120  | 120 | ●      | Pro-Cup - Hard        | ✓     |       | ✓          |
|  | 1810/30            | 30  | 1810/120 | 120 | ●      | Pro-Cup - Soft        |       | ✓     | ✓          |
|  | 1811/30            | 30  | 1811/120 | 120 | ●      | Pro-Cup - Hard        |       | ✓     | ✓          |
|  | 992/30             | 30  | 992/120  | 120 | ●      | Pro-Cup Junior - Soft | ✓     |       | ✓          |
|  | 993/30             | 30  | 993/120  | 120 | ●      | Pro-Cup Junior - Hard | ✓     |       | ✓          |
|  | 1812/30            | 30  | 1812/120 | 120 | ●      | Pro-Cup Junior - Soft |       | ✓     | ✓          |
|  | 1813/30            | 30  |          |     | ●      | Pro-Cup Junior - Hard |       | ✓     | ✓          |



### Kerr's paste carriers

Available as single use or multiple use, these ergonomic prophy paste carriers are easy to fit to your finger, and have enough space/volume even for bigger amounts of paste (when/if needed).

#### Ring Pots - Not autoclavable

| Part No. | Description                                |
|----------|--|
| 985      | Ring Pots – Blue – Pack of 100             |
| 986      | Ring Pots – Assorted Colours – Pack of 100 |

#### Prophy Clips - Autoclavable

| Part No. | Description              |
|----------|--------------------------|
| 3151     | Prophy Clips – Pack of 6 |