Caps Dispenser

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
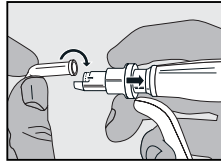
The parallel lever movement allows a simple, precise application, of even highly filled composites.

Application:


1. Pull the quick-release fastener all the way back.

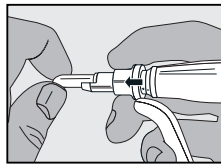
2. Insert the cap into the dispenser and slowly release the fastener until the cap is securely locked into place.

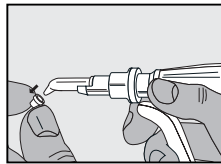3. Rotate the tip of the cap to the desired direction and remove the protective cap. Extrude the material by slowly and evenly pressing the handle and lever together, applying the material directly into the cavity. Do not apply too much pressure.

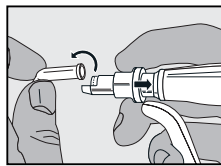4. After the material has been applied, pull the quick-release fastener back and pull the cap forward and up to remove it.

5. Clean and disinfect the **Caps Dispenser** after use. The **Caps Dispenser** can be steam-sterilized up to 134°C (steam autoclave, follow the manufacturer's instructions for use). In addition, ethanol and propanol can be used for cleaning.

Additional Information:

The Caps Dispenser system allows quick and hygienic direct application. The material can be precisely dispensed into the cavity through the small opening of the cap. Detailed information for each of the **VOCO** preparations can be found in the individual instructions for use. Do not use visibly damaged devices.

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