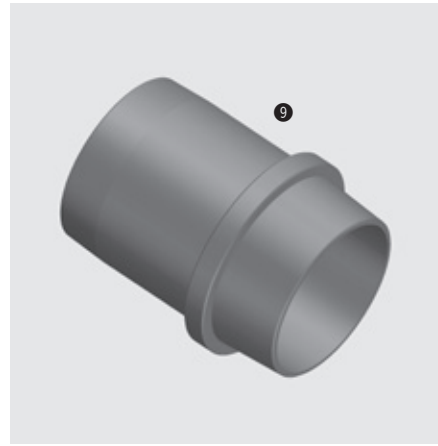
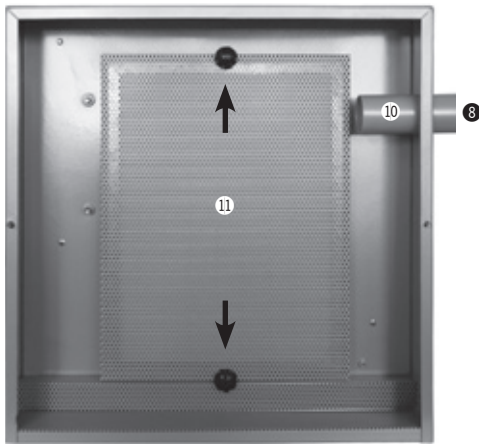
 **ceramill airstream**

**Bedienungsanleitung**  
**User Manual**  
**Mode d'emploi**  
**Istruzioni d'uso**  
**Modo de empleo**  
**Upute za uporabu**

**AMANNGIRRBACH**

Deutsch	04 - 08
English	09 - 13
Français	14 - 18
Italiano	19 - 23
Español	24 - 28
Hrvatski	29 - 32

## CERAMILL AIRSTREAM

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## CERAMILL AIRSTREAM

- ① On/Off switch
- ② Adjustable extraction control
- ③ Re-set button (safety switch)
- ④ Handle
- ⑤ Flap for changing the filter bag/  
Micro filter
- ⑥ Socket for control cable (for automatic  
operation in the Ceramill Motion)
- ⑦ Mains cable socket
- ⑧ Point of connection for extraction tube
- ⑨ Suction adapter (Art. No. 179601) for connec-  
tion to the required unit (not included in the  
delivery contents of the Airstream)
- ⑩ Extraction support for filter bag
- ⑪ Micro filter
- ⑫ Flap for changing the carbon brushes

### PRODUCT DESCRIPTION

The extraction unit Ceramill Airstream is suitable for the following devices from Amann Girrbach:

- \_ Ceramill Motion (suction adapter Art. No. 179601 included in the delivery contents)
- \_ Ceramill Multi-x/Base
- \_ Smartbox Invest

Likewise, this extraction unit can also be used with every device previously listed in Amann Girrbach's product range (e.g. Diacut Vairo, Multitrim, Milltrim). The separate mains cable attachment allows automatic operation with the CNC milling machine Ceramill Motion.

The Ceramill Airstream has a special filter bag and a HEPA micro filter, which filters up to 99.97 % of fine dust particles (i.e. zirconium oxide dust particles) and is classified in the fine dust particle category M.

### INITIAL OPERATION

After unpacking the unit, please check for possible transport damage. Should you find any damage, please claim the damage from your supplier immediately. The Ceramill Airstream is available in 2 voltage strengths (100-120V and 230V). Before connecting to a power source, please check whether the voltage on the type plate at the back of the housing corresponds to your voltage, and make sure the ON/OFF switch is OFF.

### TUBE CONNECTION

Connect the extraction tube at the designated connection point ⑦ directly to the extraction unit and the conical side to the chosen device by means of the supplied adapter ⑨.

### ELECTRICAL CONNECTION

#### Manual Operation

(e.g. for Smartbox Invest):

Connect the extraction ⑦ to the power source using the enclosed mains cable. The extraction unit is put into operation using the ON/OFF switch ① at the positions ON/OFF.

### Half-automatic operation

(e.g. Ceramill Multi-x/Base):

Connect the extraction 7 to the chosen device by connecting the mains cable to the connection cable (supplied with the Ceramill Multi-x/Base). Turn the ON/OFF switch onto position ON. The extraction unit is operated once the device, to which the extraction unit is connected, is switched on.

### Automatic operating (e.g. Ceramill Motion)

Connect the extraction 7 to the power source with the enclosed mains cable. Plug the control cable (e.g. from the Ceramill Motion) into the control cable socket 6 in the extraction unit. Switch the ON/OFF switch to AUTO.

## MAINTENANCE

The Ceramill Airstream comprises a filter bag and a HEPA micro filter. NEITHER filter is washable! The Ceramill Airstream must never be used without both filters being present. Furthermore, only original filters may be used, otherwise the warranty claim will become invalid, and sufficient extraction of fine dust particles (according to dust particle category M) cannot be guaranteed.

Please ensure of sufficient respiratory protection. To change the filters, loosen both black screws on the right hand side of the extraction unit and remove the flap 5.

### 1. To change the filter bag

After opening the flap, simply remove the filter bag from the extraction supports 10 and place a new filter bag (item no. 178610) on top.

### 2. To change the HEPA micro filter

The micro filter 11 is located under the filter bag. To change, loosen both black screws, remove the lattice attachment and exchange the micro filter (item no.178611). It is recommended that the HEPA microfilter be changed at the latest after 500 operating hours.

### 3. Replacing the carbon brushes

(Only relevant with units, which do not have a long life motor installed - units up to the E series)

It is recommended that the carbon brushes (Art. No. 178614) be changed after 500 operating hours. The brushes should only be replaced by authorised personnel (electrician or technical service).

If the carbon brushes are used up and have not been replaced, the motor stops for reasons of safety. Once the carbon brushes have been replaced, the Ceramill Airstream is operational for the next 500 operating hours.

It is essential to remove the mains plug before changing the carbon brushes!!!

- a) Remove all hoses and cables from the Ceramill Airstream.
- b) Loosen the screws on the left side of the Ceramill Airstream 12 using a screwdriver, remove the flap and foam cover.
- c) There is one carbon brush each at the right and left side of the turbine that are retained with 2 screws. Loosen the screws, lift the mounting bracket, lift the back of the carbon brush and pull out towards the back.

- d)** Insert the replacement carbon brush upright with the guide pin facing downwards into the mounting bracket until the guide pin is fixed in position.
- e)** Tighten the screws, place on the foam cover, screw down the flap tightly, connect the mains plug.
- f)** Run the Ceramill Airstream for 20 min. on the lowest level without connecting it to a unit (e.g. Ceramill Motion). This is to adapt the carbon brushes.
- g)** Connect the Ceramill Airstream to the unit with which it is intended to operate.

#### CAUTION!

As with all electronic devices, when dealing with this extraction equipment, it is always essential to handle with care and attention. The unit must be serviced and maintained regularly, in order to guarantee smooth, trouble-free operation. If necessary maintenance work is not undertaken or worn parts are not replaced, it could jeopardise the safety of the unit. In this case, the manufacturer assumes no liability whatsoever. The extraction unit is designed so that it can be connected to a power source continuously.

#### 4. Replacing the motor

If the motor fails, after approx. 2,500 operating hours, it must be replaced. All units from the F series onwards have a long life motor, with which the carbon brushes no longer have to be exchanged. The long life motor is also available as a replacement motor for units up to the E series, which until now have not been fitted with a long life motor.

The long life motor is identified by a sticker on the unit. The type of motor which is in the unit can be set in the Ceramill Motion software. Please contact the Ceramill helpdesk for information regarding this.

The motor should only be replaced by an electrician or technical service. Please contact your technical service.

### Technical Data:

D/W/H:	406 x 280 x 423 mm
Weight:	16 kg
Electrical connection values:	230 V/3,6 A 110-120 V/7,0 A
Hertz:	230V/50 Hz 110-120 V 60 Hz
Performance:	1000 Watt
Noise:	52 dbA
Bag capacity:	10 l
Hose - diameter/Length:	38 mm/1,8 m
Adapter diameter:	37 - 38 mm
	HEPA micro filter (99,97 %), Filter class H12, dust class M

### Scope of delivery:

Art.-Nr.	178600: 230 V
	178600V100-V120: 100-120 V
	1 Filter bag, 1 microfilter, 1 suction hose
	1.8 m, 1 operating instructions

### Accessories/Components:

Art.Nr.:	
178601	Airstream suction adapter (suitable for use with the Ceramill Motion/Base, Ceramill Multi-x, Smartbox Invest)
178610	Airstream filter bag (5 items)
178611	Airstream micro filter (1 item)
178613	Airstream extraction hose
178614	Airstream replacement carbon brushes with fuse (2 brushes)
178615	Airstream long life motor 230V (up to the E series)
178615	V100-V120 Airstream long life motor 100V-120V (up to the E series)
178616	Airstream long life motor 230V (from the F series onwards)
178616	V100-V120 Airstream long life motor 100V-120V (from the F series onwards)

<b>!</b>	<p><b>WARNING AND SAFETY INSTRUCTIONS:</b></p> <p>Ceramill Airstream was developed to be used in the dental laboratory. Initial and further operation are to be in line with the operating instructions. The manufacturer does not take responsibility for any damages that result from any other application or inappropriate handling. In addition, the user is obliged to use the device on his own responsibility and to see before use whether the device is appropriate for the intended purpose unless this is defined in the operating instructions.</p> <ul style="list-style-type: none"> <li>_ <b>Please do not vacuum sharp objects!</b></li> <li>_ <b>Please keep off "extraction" always, before your take out the mains plug. Otherwise extraction may be damaged - in the event</b></li> </ul>	<p><b>of which the manufacturer does not accept any liability.</b></p> <ul style="list-style-type: none"> <li>_ <b>Do not operate the extraction in a small closed room and always keep enough distance to the wall.</b></li> <li>_ <b>The ventilator on the back has to be kept clear at all times.</b></li> <li>_ <b>Keep the device clean.</b></li> <li>_ <b>Exchange filter bag and micro filter.</b></li> <li>_ <b>When changing the filter observe sufficient breathing protection and take the mains plug out beforehand!</b></li> <li>_ <b>Keep clear the hoses.</b></li> <li>_ <b>Protect device against humidity always.</b></li> <li>_ <b>No contact with water or other liquidity!</b></li> <li>_ <b>Please contact technical services in the event of motor or any other defects.</b></li> <li>_ <b>Always connect to grounded circuits!</b></li> </ul>
----------	--	--

Issue	Possible Cause	Solution
Extraction does not start	<ul style="list-style-type: none"> <li>_ Mains plug not inserted</li> <li>_ ON/OFF not switched ON</li> <li>_ Fuse jumped out</li> <li>_ Motor burnt out</li> </ul>	<ul style="list-style-type: none"> <li>_ Examine mains plug</li> <li>_ Check ON/OFF switch</li> <li>_ Press reset button ③ at device</li> <li>_ Call technical service</li> </ul>
Extraction extremely loud	<ul style="list-style-type: none"> <li>_ Turbine has contact with case</li> </ul>	<ul style="list-style-type: none"> <li>_ Call technical service</li> </ul>
Insufficient extraction performance	<ul style="list-style-type: none"> <li>_ Obstacle in system</li> <li>_ Filter full</li> </ul>	<ul style="list-style-type: none"> <li>_ Remove obstacle</li> <li>_ Exchange filter</li> </ul>
Motor switches off	<ul style="list-style-type: none"> <li>_ Power consumption too high</li> <li>_ Carbon brushes are used</li> <li>_ Motor burnt out</li> </ul>	<ul style="list-style-type: none"> <li>_ Check if filter is placed properly, press reset button</li> <li>_ Call technical service</li> <li>_ Call technical service</li> </ul>
Air flow extremely strong	<ul style="list-style-type: none"> <li>_ Filter not positioned correctly</li> </ul>	<ul style="list-style-type: none"> <li>_ Examine filter</li> </ul>

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**Amann Girrbach AG**
**Amann Girrbach GmbH**