CLEARFIL™ DC Activator

EN	ENGLISH	English	DENTAL BONDING AGENT
FR	FRANÇAIS	French	AGENT DE LIAISON DENTAIRE
ES	ESPAÑOL	Spanish	AGENTE DE ADHESIÓN DE USO DENTAL
IT	ITALIANO	Italian	AGENTE DI ADESIONE DENTALE
NL	NEDERLANDS	Dutch	DENTALE BONDING
DE	DEUTSCH	German	Dentaler Haftvermittler
SV	SVENSKA	Swedish	DENTALT BONDINGSMATERIAL
NO	NORSK	Norwegian	DENTALT BINDINGSMIDDEL
FI	SUOMI	Finnish	HAMPAAN SIDOSAINE
DA	DANSK	Danish	DENTALT BONDINGMATERIALE
PT	PORTUGUÊS	Portuguese	AGENTE DE ADESÃO DENTÁRIO
EL	ΕΛΛΗΝΙΚΑ	Greek	ΟΔΟΝΤΙΚΟΣ ΣΥΓΚΟΛΛΗΤΙΚΟΣ ΠΑΡΑΓΟΝΤΑΣ
TR	TÜRKÇE	Turkish	DENTAL BONDING AJAN
PL	POLSKI	Polish	STOMATOLOGICZNY SYSTEM WIĄŻĄCY
RO	ROMÂNĂ	Romanian	AGENT DENTAR ADEZIV
HR	HRVATSKI	Croatian	DENTALNI ADHEZIV
HU	MAGYAR	Hungarian	FOGÁSZATI KÖTŐANYAG
SL	SLOVENŠČINA	Slovenian	DENTALNO VEZIVNO SREDSTVO
CS	ČESKY	Czech	DENTÁLNÍ POJIVO
SK	SLOVENSKY	Slovak	DENTÁLNE TMELIACE ČINIDLO
BG	БЪЛГАРСКИ	Bulgarian	СТОМАТОЛОГИЧЕН БОНДИНГ АГЕНТ
UK	УКРАЇНСЬКА	Ukrainian	СТОМАТОЛОГІЧНИЙ АДГЕЗИВ
ET	EESTI	Estonian	DENTAALNE ADHESIIV
LV	LATVIEŠU	Latvian	ZOBĀRSTNIECĪBAS SAISTVIELA
LT	LIETUVIŠKAI	Lithuanian	ODONTOLOGINIS RIŠIKLIS

Kuraray Noritake Dental Inc.

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

EMERGO EUROPE

Prinsessegracht 20
2514 AP The Hague, The Netherlands

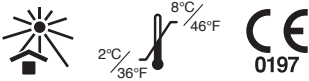
Kuraray Europe GmbH

Philipp-Reis-Str. 4,
65795 Hattersheim am Main, Germany
Phone:+49 (0)69 305 35835 Fax:+49 (0)69 305 98 35835
URL:<https://www.kuraraynoritake.eu>

CLEARFIL™ DC Activator

I. INTRODUCTION

CLEARFIL DC Activator activates the dual-curing mechanism of the CLEARFIL BONDING SYSTEM (e.g. CLEARFIL SE BOND 2 or CLEARFIL SE Protect). The product is mixed with the BOND and can be used with dual-cure or self-cure composite filling materials, cements, or core build-up materials.

II. INDICATIONS

When used with the CLEARFIL BONDING SYSTEM, the product is indicated for the following uses:

- [1] Core build-up in conjunction with self- or dual-cured core build-up materials
- [2] Cementation in conjunction with self- or dual-cured composite resin cements

III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to methacrylate monomers

IV. PRECAUTIONS

1. Safety precautions

1. This product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components.
2. If the patient demonstrates a hypersensitivity reaction such as rash, eczema, features of inflammation, ulcer, swelling, itching, or numbness, discontinue use of the product and seek medical attention.
3. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect them in the event of splashing material.
4. If the product comes in contact with human body tissues, take the following actions:
 - <If the product gets in the eye>
 - Immediately wash the eye with copious amounts of water and consult a physician.
 - <If the product comes in contact with the skin or the oral mucosa>
 - Immediately wipe the area with a cotton pellet or a gauze pad moistened with alcohol, and rinse with copious amounts of water.
5. Use caution to prevent the patient from accidentally swallowing the product.
6. Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers or any other components.
7. Dispose of this product as a medical waste to prevent infection.

2. Handling and manipulation precautions

1. The product must not be used for any purposes other than specified in [II.INDICATIONS].
2. The use of this product is restricted to licensed dental professionals.
3. Clean the cavity sufficiently to prevent poor bonding. If the adherend surface is contaminated with saliva or blood, wash it thoroughly and dry before bonding.
4. The product contains ethanol. Dry the entire adherend surface sufficiently by blowing mild air for more than 5 seconds until the mixture does not move. Adjust the air pressure according to the shape and size of the cavity and the prosthetic appliance. Use a vacuum aspirator to prevent the bond liquid from scattering.
5. After mixing the product with a bonding agent, use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 90 seconds after dispensing.
6. Do not use CLEARFIL DC Activator in conjunction with other bonding agents except those from the CLEARFIL BONDING SYSTEM.

3. Storage precautions

1. The product must be used by the expiration date indicated on the package.
2. The product must be refrigerated (2-8°C / 36-46°F) when not in use, and should be brought to room temperature for more than 15 minutes before using. After taking out of the refrigerator, the product must be left standing until it comes to room temperature.
3. The container should be tightly capped immediately after use.
4. The product must be kept away from extreme heat, direct sunlight or a flame.
5. The product must be stored in a proper place where only dental practitioners can access.

V. COMPONENTS AND INGREDIENTS

- 1) Components
Please see the outside of the package for contents and quantity.
- 2) Ingredients
Principal ingredients
Ethanol, Catalysts, Accelerators

VI. CLINICAL PROCEDURES

CLEARFIL DC Activator can be used in combination with either CLEARFIL SE BOND 2 or CLEARFIL SE Protect. When using in conjunction with another CLEARFIL BONDING SYSTEM, refer to each specific bonding agent's instructions. Do not mix with any other adhesive not recommended in the Instructions for Use for CLEARFIL DC Activator.

A. Standard procedure for core build-up

A-1. Isolation, moisture control, preparing root canal, post surface treatment, and acid-etching of tooth

Refer to the instructions for use of the core material or composite resin cement used.

A-2. Pretreatment of tooth

Apply the PRIMER of CLEARFIL SE BOND 2 or CLEARFIL SE Protect to the adherend surface, according to the manufacturer's instructions.

A-3. Bonding to tooth

1. Dispense one each drop of the product and the BOND of CLEARFIL SE BOND 2 or CLEARFIL SE Protect into a well of the dispensing dish and mix immediately before application.

[CAUTION]

Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 90 seconds after dispensing.

2. Apply the mixture to the entire cavity wall with the applicator brush. Use caution not to allow saliva or exudate to contact the treated surfaces.

3. Dry the entire adherend surface thoroughly by blowing mild air for more than 5 seconds, until the bond does not move. Use a vacuum aspirator to prevent the bond liquid from scattering. Remove the excess mixture with a paper point. After wiping away the excess mixture, dry the adherend surface again, if necessary.

[NOTE]

To dry thoroughly, adjust the air pressure according to the shape and size of the adherend surface.

4. Light-cure is recommended for optimum performance. Regarding the light-curing time, refer to the instructions for use of the bonding agent used.

A-4. Placing the post and core build-up

Place the post and core build-up using a core material according to the manufacturer's instructions.

B. Standard procedure for cementation

B-1. Preparation of the cavity, prosthetic restorations

Refer to the instructions for use of the composite resin cement used.

B-2. Pretreatment of tooth

Refer to section A-2.

B-3. Bonding to tooth

Mix with the BOND, and apply it to the adherend surface referring to section A-3.

B-4. Cementing

Cement the prosthetic restoration using composite resin cement according to the manufacturer's instructions for use.

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

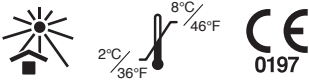
[NOTE]

If a serious accident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.

[NOTE]

"CLEARFIL" and "SE BOND" are trademarks of KURARAY CO., LTD.

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„CLEARFIL“ ir „SE BOND“ yra registruoti KURARAY CO., LTD. prekių ženklai.