G² Concept

Original

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We are pleased that you decided to buy a highly developed piece of equipment from SCHICK and would like to wish you every success when working with your new G2 Concept all purpose model saw.

We wrote these instructions to enable you to get accustomed to your new piece of equipment and to provide you with the correct operating and maintenance instructions.

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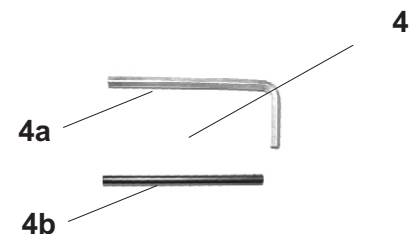
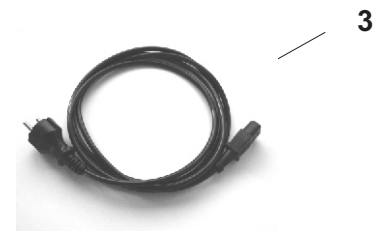
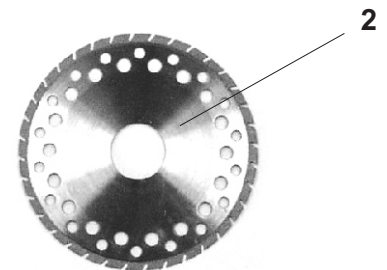
Index	Page:
1. List of contents - accessories	4
2. Range of applications	5
3. General information Caution !	6
4. Picture G2 Concept - operating elements	7
5. Setting up	8
6. Operating the unit	9
7. Exchanging the saw blade	10
8. Calibrating laser beam	11
9. Maintenance and care	12
10. Possible faults	12
11. Technical data	13
12. Declaration of conformity	14

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1. List of contents - accessories/spare parts

G2 Concept art.no.: 1500/5
 Model saw complete with line marking laser (1)
 diamond saw blade (2)
 mains cable (3)
 tools for changing saw blade (2 pcs.) (4)

optional accessories & spare parts	art.-no.
adapter for suction (6)	1580
diamond saw blade (2)	1967/2
mains cable (3)	2160
hexagon socket wrench	
SW 4 (4a)	W602000400
SW 3 (page 11)	W602000300
SW 2,5 (page 11)	1588
bolt Ø4 mm (4b)	1934
fuse 800 mA/T 220V(2A/T100V-115V)	3105 (3106)
fuse for motor 8 A T	1932

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Laserklasse: 3R

2. Range of Applications

Until now, many models have necessitated saw cutting by hand, due to the limited space between prepared teeth or irregularities in the alignment of the plaster dies. With the new G2 Concept, these sawing operations can be carried out mechanically with highest precision, because the G2 Concept is operating from below. The G2 Concept provides for the optimum saw cutting of all models without damaging the preparation lines.

Soft starting, instant stopping and two-hand sensor technology are guarantee for your.

Conditions of environment: - interior 10° - 40°C

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WEEE-Reg.-Nr. DE 78620387

3. General Information

- 3.1 Ascertain that your mains supply coincides with the data in the rating plate.
- 3.2 The model saw G2 Concept is not suitable for the following applications:
 - in areas where there is a risk of explosion
 - on patients
 - for working on moist materials
- 3.3 Insure that all regulatory requirements are observed during use
 - always wear protective glasses
 - take care for good illumination
 - use dust suction

Caution

- Use only with original SCHICK saw blade
- Repairs and other technical procedures must only be carried out by suitable qualified personnel, authorized by SCHICK.
- SCHICK do not guarantee the model saw G2 Concept should it not have been used in accordance with the instructions.

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
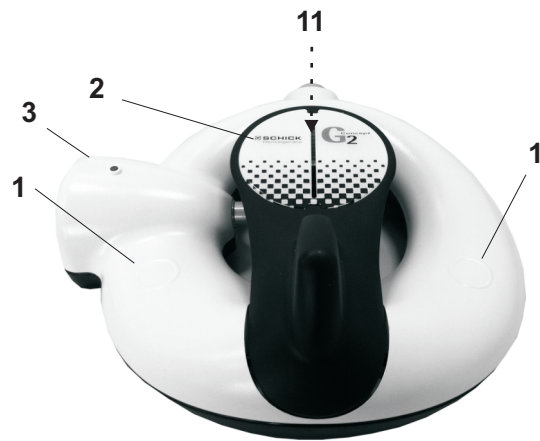
4. Picture G2 Concept

- operating elements

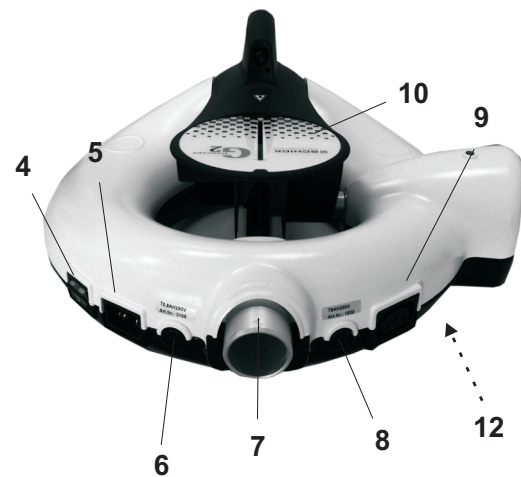
- 1 - hand rest
- 2 - saw table
- 3 - control lamp
- 4 - mains switch
- 5 - socket for mains cable
- 6 - fuse 800 mA T/230V (2A T/100V/115V)
- 7 - extractor socket for external dust suction
- 8 - fuse motor 8 A T
- 9 - socket to plug in external dust suction
- 10 - line-marking laser
- 11 - saw blade

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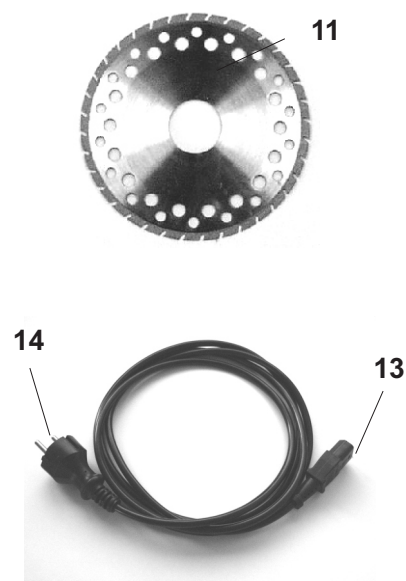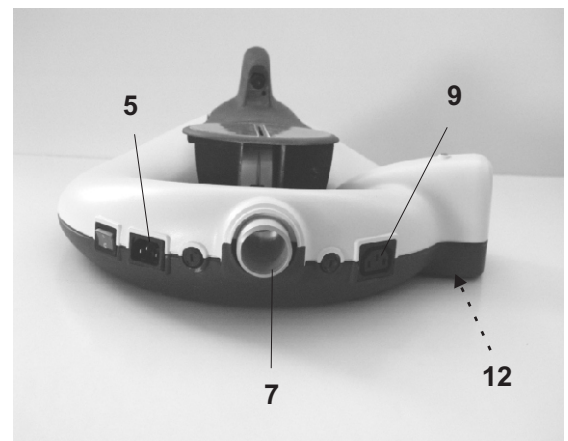
5. Setting up

Carefully unpack the contents of the G2-Concept saw, inspect for any damages and that all components are present. Remove foam shipping block from model table that protects blade during shipping. Confirm voltage information on identification place on bottom of the G2-Concepts saw and verify you have received the proper power cable for your electrical needs. Proceed to plug your power cable into socket (5), if you have purchased the optional suction on/off cable proceed to plug this cable into suction socket (9) (note: your vacuum/suction machine will plug into the female end of this cable and the G2-Concept saw will control the on/off of your vacuum/suction).

Be sure keep your box and packaging material for future use

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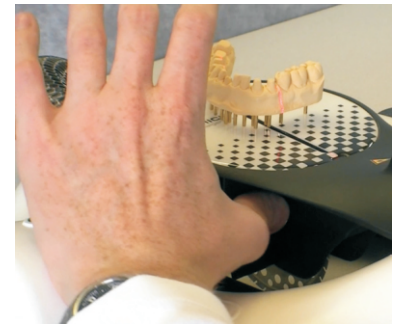
6. Operating the unit

Turn on device at the main switch in the back of the G2 Concept.

To be sawed press the model with the pins downwards.

Put model on the saw table and position with both hands.

Pay attention that both wrists lie on the sensors and touch them exactly to activate them thereby. Carefully, slowly contacting does not activate the sensors! Take thumb under the saw table and fix the model with forefinger and middle finger on the model table. Align position in the laser line accurately. Press model easily downwards. The saw and the external suction will be turned on automatically in operation by the inserted sensor technology. Press model under light pressure farther downwards. Now the saw blade dives from below by the supporting plate and into the plaster arch and produces a clean saw cut in parallel to the pins. If the optimal cut depth is reached, drive back the saw table slowly again in the starting position. Then the saw switches off itself again automatically. To work nearly dust free the external suction will run a few time longer and stop then automatically.

Security advice

Operate saw never if hand or finger are in the area of the laser line! Risk of injury! Saw operate only with protection eyeglasses! Saw operate absolutely with corresponding dust suction!

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
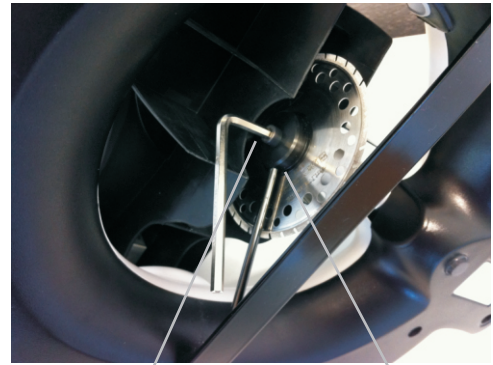
B

7. Change of the saw blade

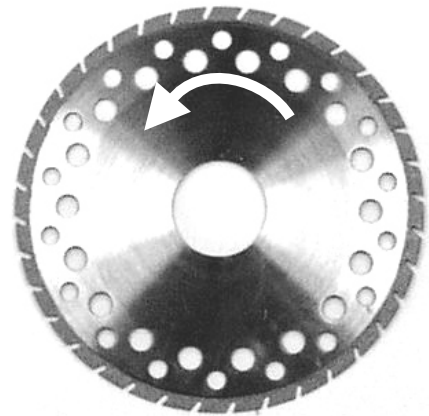Pull net plugs.
Decrease suction tubes.
Turn G2 Concept and place preferably on the high thighs.
Unscrew inside hexagonal screw (A) and remove fixation ring (use counter pressure with the provided tool in the intended whole (B) on the shaft) .
Exchange saw blade (Use only original SCHICK diamond discs!)
Insert fixation disc again and scew in the inside hexagonal screw under counterholds with the intended tool.

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8. Calibrating the laser beam of modelsaw

G2 - Concept (as of series number 150.416)

1. Switch the unit on.

Use allen key 3mm (pic.1) horizontally and undo clamp bolt.

ATTENTION: Avoid direct eye contact with the laser beam!

3. Use allen key 2,5mm with an ball-head sloping (pic.2) into levelling screw placed below the clamp bolt.
4. When screwing in the levelling screw (pic.2) the laser beam moves to the right, when screwing out the levelling screw the laser beam moves to the left.
5. Adjust laser beam in the desired position and secure clamp bolt (pic.1) by using allen key 3mm.

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Attention
Do not look directly
into the laser beam source!

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f.e.: motion of laser
beam to the right side

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9. Maintenance and care

Clean device occasionally with a soft cloth.

Further maintenance is not necessary!

Never use compressed air or cleaning agent!

10. Possible faults

Net switch lighting and net LED are not working:
- > net fuse (800 mA/T No:3105 230V ; 2A/T No:3106 100V/115V) - check and exchange when required!

Engine does not start:

- > See by net fuse. If in order, then check fuse for engine (8 A/T No. In 1932) and exchange when required.

By all other faults send the device directly to SCHICK-Dental or an authorized dealer.

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11. Technical Data

Dimensions:	Width: 375mm Height: 160mm Depth: 390mm
Weight:	4,5 kg
Power supply:	230 / 115 / 100 V
Nominal frequency:	50/60 Hz
Power input:	130 W
Speed range:	7.500 min-1
Diamond saw blade: segmented	Ø 85 x 20 mm
Width of cut:	0,3 mm
depth of cut:	26,5mm
Laser class:	3R
Safety class:	2

This unit complies with the current VDE (German association of electrical technicians) regulations concerning safety and suppression.

These instructions should be readily accessible and are best kept close to the unit itself.

We would like to take this opportunity to advise you that a proper repair service and suitable qualified personnel are required for such highly developed technical equipment.

SCHICK guarantees to carry out perfect repairs using original spare parts and we are extending another 6 month guarantee on repairs in our house.

Subject to technical modification without prior notice

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We, SCHICK GmbH
Schick GmbH