

MATRIX MTA+

Dosing block MTA+



BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

INTENDED USE

MTA + dosing block is designated for forming precise doses of MTA+ product for the tooth cavity or root canal.


INSTRUCTIONS FOR USE

Prepare MTA+ product as per package leaflet supplied. On each side of the dosing block there are grooves of different sized diameters ranging from 0,7 mm to 1,00 mm.


Place the product on one side of the dosing block so that the product fills the chosen groove.

With the material formed and the desired dose selected, should be taken from the dosing block by means of a sterile instrument.

PICTURES

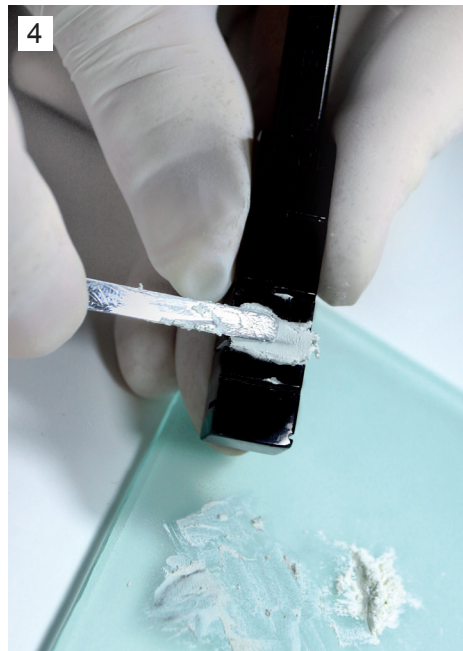

5

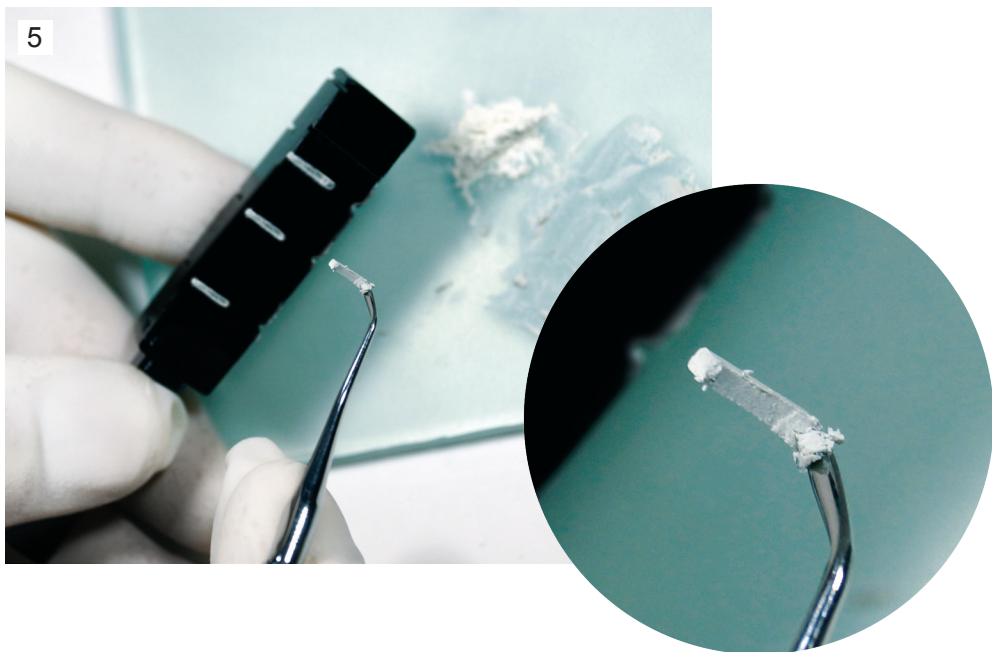

WARNING

The product should be used within 10 minutes after preparing for application.
Do not leave the product in the dosing block.

PRECAUTIONS

Keep the instructions enclosed in the package leaflet of MTA+ product for correct future use.

STORAGE

Keep out of the reach of children.

MAINTENANCE

Before use: wipe with isopropyl or ethyl alcohol.

After use: remove the remains of MTA+ product using a brush and water with detergent. Clean the dosing block by wiping it with isopropyl or ethyl alcohol.

Steam sterilization of the device in the temperature of 121°C doesn't cause any change of its physical characteristics (shape, dimension).



FOLLOW THE INSTRUCTION



PPH CERKAMED Wojciech Pawłowski
37-450 Stalowa Wola, Polska
ul. Kwiatkowskiego 1
Tel/fax 00 48 15 842 35 85