- en Instructions for Use
- de Gebrauchshinweise
- fr Indications d'emploi
- it Istruzioni per l'uso
- es Instrucciones de uso
- pt Instruções de Utilização
- nl Gebruiksaanwijzing
- sv Bruksanvisning
- da Brugsvejledning
- nb Bruksanvisning
- fi Käyttöohjeet
- lv Lietošanas instrukcija
- pl Instrukcja użytkowania
- tr Kullanım Endikasyonları
- cs Návod k použití
- el Οδηγίες χρήσης
- ro Instrucțiuni de utilizare
- ru Показания к применению средство

Doxa

Contact

If you have any questions, please contact:

 Doxa Dental AB	For US and Canada customers:
Finvids väg 8-10	Directa Inc.
SE-194 47 Upplands Väsby	1.800.537-8765
SWEDEN	www.ceramirUS.com

Phone: +46 (0)8 506 505 75

www.ceramir.se

2460

© Doxa Dental AB 2023. All rights reserved.

ENGLISH

CAUTION FOR PROFESSIONAL DENTAL USE IN RECOMMENDED INDICATIONS ONLY

1. PRODUCT DESCRIPTION

Ceramir® Crown & Bridge is a permanent, radiopaque, bioceramic luting cement. The main component in the powder is calcium aluminate and in the liquid, water. The cement, which hardens as a result of a chemical reaction, is self-setting. In Ceramir Crown & Bridge QuikMix, the powder is supplied separately in a jar and the liquid is supplied in a drop bottle. The minimum net mass of powder is 15g and the minimum net volume of liquid is 10 g (9 ml).

ACCESSORIES:

A scoop for powder dosing is included with powder

A non-absorbing mixing pad

A plastic spatula for cement mixing

2. INDICATIONS FOR USE

Ceramir Crown & Bridge is intended for permanent cementation of:

- Metal and porcelain fused to metal crowns and bridges
- Gold inlays and onlays
- Cast or prefabricated metal posts
- High-strength ceramic crowns and bridges suitable for conventional cementation (e. g. zirconia, alumina, and lithium disilicate)

Contraindications

- Do not use as a direct pulp-capping agent. In case of close proximity to the pulp, use accepted standard clinical procedures.

Precautions

- Ceramir Crown & Bridge contains polyacrylic acid, which may be irritating to eyes and skin. Avoid contact with eyes. In case of skin contact, rinse with plenty of water. In case of contact with eyes, rinse immediately with plenty of water and seek medical care.

- Do not use on patients with known allergy to polyacrylic acid. In rare cases, the product may cause sensitivity to some persons. If such reactions are observed, discontinue the use of the product and consult a physician.

3. STEP-BY-STEP INSTRUCTIONS

ATTENTION DO NOT USE ANY PRE-TREATMENT AGENTS, E.G. SILANES, PRIMERS OR BONDING AGENTS WHEN CEMENTING WITH CERAMIR CROWN & BRIDGE. USE OF ANY SUCH PRE-TREATMENTS MAY COMPROMISE THE BONDING TO THE CEMENT

3.1 Tooth preparation

- Remove the temporary restoration and cement.
- If needed, clean the tooth using pumice or prophylaxis paste (with or without fluoride).
- Rinse with water.
- Air dry gently with oil free air or cotton pellets. Do not desiccate.
- Isolate the site with cotton rolls.

3.2 Preparation of the restoration

Lithium Disilicate

Etch with HF acid according to manufacturer's instructions. Do not use any pre-treatment agents, e.g. silanes, primers or bonding agents.

All other restorations

Do not use any pre-treatment agents, e.g. silanes, primers or bonding agents when cementing with Ceramir Crown & Bridge. Use of any such pre-treatments will destroy the bond between cement and restoration.

3.3 Pulp protection

- If pulp protection is required, use accepted standard clinical procedures.

3.4 Powder-liquid dispensing and mixing

The standard powder to liquid ratio is 1.8/1.0 (w/w) (1 level scoop of powder to 2 drops of liquid). Mixing pad type: Non-absorbing mixing pad or glass slab

- Dispense the desired number of level powder scoops, using the scoop provided, onto a mixing pad/slab.
- Hold the liquid bottle vertically and squeeze gently to dispense liquid (2 drops per powder scoop). For more consistent results, make sure that there is no air in the nozzle while dispensing e.g. by discarding the first drop or gently shaking the bottle while upside down.
- Close bottles immediately after use.
- Add all the powder to the liquid using a plastic spatula.
- Mix to a homogenous paste (maximum 30 seconds).

Working and setting times

- Working time (from start of mixing) at 23°C/73°F: 2 minutes.
- Setting time (from start of mixing): 3 to 8 minutes.

NOTE: ROOM TEMPERATURES ABOVE 23°C/73°F WILL ACCELERATE SETTING AND REDUCE WORKING TIME. WORKING TIME CAN BE LENGTHENED BY USING REFRIGERATED COMPONENTS OR BY MIXING ON A COLD GLASS SLAB.

3.5 Luting and finishing

- Load the mixed cement into the crown.
- Seat the restoration and stabilize it under pressure, to avoid any movements, until the cement reaches a rubber-like consistency (approx. 2 min).
- Remove excess material when the cement has reached the rubber-like consistency.
- Keep the restoration stabilized for an additional 4 minutes.
- Check the area around the margins of the restoration (especially the gingival sulcus) and remove any remaining cement.
- Advise the patient not to load their restoration for the first hour after cementation.

4. STORAGE AND SHELF LIFE

- Keep Ceramir Crown & Bridge QuikMix at temperatures between 4°C/39°F and 20°C/68°F.
- Store with jar caps securely tightened and away from high humidity.
- Do not use Ceramir Crown & Bridge QuikMix after the expiry date.

Warranty

Doxa Dental AB warrants that the Product will be free from defects in material and manufacture. This warranty shall (i) only apply within the Product's shelf life/expiry date (the "Warranty Period") and (ii) further only if the Product has been used strictly in accordance with these Instructions for Use. DOXA DENTAL AB MAKES NO OTHER WARRANTIES, EXPRESSED OR IMPLIED, REGARDING THE PRODUCT, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR NON-INFRINGEMENT OF INTELLECTUAL PROPERTY RIGHTS. Users are responsible for determining the suitability of the Product for their application.

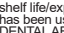
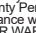
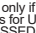
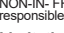
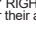
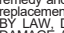
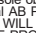
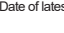
Limitation of Liability

If this Product is found defective within the Warranty Period, your exclusive remedy and Doxa Dental AB's sole obligation and liability shall be the repair or replacement of the Doxa Dental AB Product. EXCEPT WHERE PROHIBITED BY LAW, DOXA DENTAL AB WILL NOT BE LIABLE FOR ANY LOSS OR DAMAGE ARISING FROM THE PRODUCT, WHETHER DIRECT, INDIRECT, SPECIAL, PUNITIVE, INCIDENTAL OR CONSEQUENTIAL INCLUDING LOSS OF PROFITS, REGARDLESS OF THE THEORY ASSERTED, INCLUDING WARRANTY, BREACH OF CONTRACT, NEGLIGENCE OR STRICT LIABILITY.

Trademarks

Ceramir® is a trademark of Doxa Dental AB.

Date of latest revision: 2017-09-01

 Radiopaque	 Liquid	 Powder
 Scoop	 Mixing pad	 Caution
 Manufacturer	 Consult instructions for use	
 Batch code	 Catalogue number	
 Use by	 Temperature limitation	

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