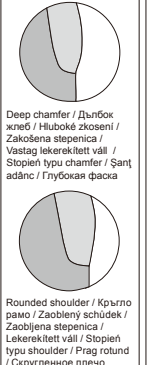
GC Initial™ LiSi Block

LITHIUM DISILIKATIN CERAMIC CAD/CAM BLOCK

For use only by a dental professional in the indications.

INDICATIONS FOR USE

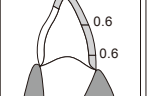
- Veneers
- Crowns
- Inlays
- Onlays
- Partial crowns
- Crown or splitted crown on top of an implant abutment

PREPARATION DESIGN


When designing restorations, the following preparation guidelines and minimum dimensions should be maintained.

- Preparation guidelines for all-ceramic restorations
- Prepare margins with deep chamfer or rounded shoulder (Fig. 1)
 - Avoid bevel margins in direct occlusal contact with the opposing tooth.
 - Antagonist contacts must be taken into consideration for the incisal edge of the preparation should be at least 1.0 mm (milling tool geometry) in order to permit optimum milling during CAD/CAM fabrication.


- Veneers (Fig. 2)
- If possible, the preparation should be located in the incisal edge of the shoulder/chamfer by 0.6 mm, and the incisal edge by 0.7 mm.
 - Reduce the cervical and/or labial area by approx. 1.2 mm.

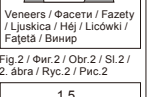- Anterior crown (Fig. 3)
- Width of the shoulder/chamfer at least 1.0 mm.
 - Reduce the incisal edge by approx. 1.5 mm.
 - Reduce the vestibular and/or oral area by approx. 1.2 mm.

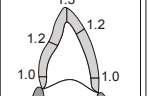- Inlay (Fig. 4)
- A preparation depth of at least 1.0 mm and an isthmus width of at least 1.0 mm must be observed in the fissure area.
 - Do not prepare undercuts.
 - Ensure that the cavity walls form an angle of 6° with the long axis of the tooth.
 - All internal edges and angles should be rounded.

- Onlay (Fig. 5)
- A preparation depth of at least 1.0 mm and an isthmus width of at least 1.0 mm must be observed in the fissure area.
 - Do not prepare undercuts.
 - Ensure that the cavity walls form an angle of 6° with the long axis of the tooth.
 - All internal edges and angles should be rounded.

- Partial crown
- Provide at least 1.5 mm of reduction in the cuspal area.
 - Width of the shoulder/chamfer should be at least 1.0 mm.

- Posterior crown (Fig. 6)
- Reduce the anatomical shape and observe the stipulated minimum thickness.
 - Width of the shoulder/chamfer should be at least 1.0 mm.
 - Reduce the incisal crown third – in the occlusal area by approx. 1.5 mm.

CAD/CAM PROCESSING

Please refer to respective instructions of the milling unit. Note: Blocks can be processed with authorized CAD/CAM software.

FINISHING AND POLISHING

Remove the finishing and polish restoration. When necessary, adjust proximal and occlusal contacts. Initial LiSi Block can be either milled and polished or milled, glazed and fired.

Polishing: Finish and polish using suitable grinding instrument, low rotation speed and light pressure. Play attention to margins and contact points when polishing the restoration. Overheating of the glass-ceramic must be avoided. Clean the restoration before cementing.

GLAZE AND STAINING (Option)

For glazing and staining, please use our dedicated GC Initial Fire Pastes NF/ONE™ and GC Initial Spectrum Stains for best results. Note: Please follow the manufacturers' instructions for use for the materials of glazing and staining.

- When firing, do not heat or cool the restorations too quickly. Rapid change in temperature could break the material. When firing, a proper furnace tray (honeycomb tray) and support pin (in combination with the GC Initial Firing Foam) should be used.

CEMENTATION

- Restoration Etching protocol
 - Apply Hydrofluoric acid gel (5-9%) for 20 seconds to clean the surfaces of the restoration.
 - Wash with water spray or an ultrasonic cleaner for 10-15 seconds.
 - Condition the etched surfaces with a silane coupling agent (such as CERAMIC PRIMER II or G-C-EM PRIMER) and allow to dry.
 - Hydrofluoric acid is for extraoral use only and must not be applied in the oral cavity.

- Cementing
 - Cement with an adhesive resin cement (such as G-C-EM LinkForce) or a self-adhesive resin cement (such as G-C-EM ONE).
 - Prior to using CERAMIC PRIMER II, G-Multi PRIMER or G-C-EM ONE, refer to the respective instructions for use.
 - In case the preparation is non-retentive, an adhesive resin cement (such as G-C-EM LinkForce) or G-C-EM ONE in combination with G-C-EM ONE ADHESIVE ENHANCING PRIMER is preferred.

SHADE

- HT: (High Translucency) A1, A2, A3, B1
LT: (Low Translucency) A1, A2, A3, B1

SIZE

PACKAGE

STORAGE

CAUTION

UNDESIRED EFFECTS - Reporting:

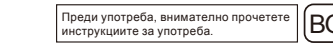
WHIMAGES

DECLARATION OF NON-INTENTIONAL EFFECTS:

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CEMENTOVÁNÍ

- Protokol leštění náhrady
 - Apply Hydrofluoric acid gel (5-9%) for 20 seconds to clean the surfaces of the restoration.
 - Wash with water spray or an ultrasonic cleaner for 10-15 seconds.
 - Condition the etched surfaces with a silane coupling agent (such as CERAMIC PRIMER II or G-C-EM PRIMER) and allow to dry.
 - Hydrofluoric acid is for extraoral use only and must not be applied in the oral cavity.

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