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EN
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY
COMPOSITION: Polyester/Polyamide yarn conjugated. Impregnated with Aluminum Chloride Hexahydrate 0.50 mg ± 0.10 mg/in.
INTENDED USE Retraction cord impregnated with Aluminum Chloride Hexahydrate. Used for the retraction of gingival tissues around tooth to improve impression results and gingival edge haemostasis.
INSTRUCTIONS FOR USE Cut the impregnated cord to the required length using the blade provided in the lid of the box. Place it tightly on the neck of the tooth and press it into the gingival sulcus using a plugger. Leave the cord in the sulcus for 5 minutes. Remove the cord immediately before taking an impression.
CONTRAINDICATIONS Do not use in patients allergic to aluminum chloride hexahydrate.
PRECAUTIONS Product for dental use only. Keep out of reach of children. This product can cause eye and skin irritation. If a serious device-related incident occurs, it must be reported to the manufacturer and to the competent authority of the State where the user or patient is domiciled.
STORAGE The expiry date is on the direct packaging. Close the lid after use and store in a dry place.
HANDLING THE PRODUCT CONTAINER Empty containers should be disposed of responsibly or returned to the manufacturer.
PACKAGING A 254 cm (100 in) dispenser.
Last update: 01.04.2021/5

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