PL YELLOW PORCELAIN ETCH

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EN YELLOW PORCELAIN ETCH

Porcelain etch

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: hydrofluoric acid 9,5%, base gel, dye

INTENDED USE

The product is used for porcelain etching during reparations of the dentures and for etching of the porcelain veneers, crowns or inlays before their cementation.

INSTRUCTIONS FOR USE

Before the treatment immerse attached applicator in alcohol (ethyl alcohol) and then fix it onto the syringe.

1. Isolate the treatment field with a rubber dam.
2. Screw the elastic blue applicator on the syringe with the etchant, replacing the safety plug with this applicator.
3. Apply a small amount of etching preparation on the prepared and dried surface. Immediately close the syringe with the safety screw.
4. Leave the etching preparation on for 60 seconds and then rinse and dry the procedure area.
5. Apply Silane according to the instruction for use provided by the manufacturer.

Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

PRECAUTIONS

The preparation is corrosive: avoid contact with the skin and mucosa membranes. In case of contact, rinse immediately with plenty of running water and contact a doctor. Avoid contact with eyes. In case of contact, rinse with plenty of water holding the eyelid wide open and seek ophthalmologist's advice. If swallowed, do not induce vomiting and contact a doctor. Use a rubber dam and protection for patient's eyes when repairing teeth in the patient's oral cavity. Use a vacuum system while etching. Equip staff with protective clothing, gloves and visors.

STORAGE

Store in a tightly closed original container below 25 °C. Keep out of reach of children. The expiry date is on the direct packaging. Once container is open, the expiry date doesn't change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Syringe of 2 ml of preparation, elastic applicators.

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