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(TI-039) ver.2 (11/2021)

Dental Zirconia

HTML PLUS (High Translucent Multi Layered PLUS)

[ENGLISH] INSTRUCTIONS FOR USE

I. Introduction

This IFU is for KATANA Zirconia HTML PLUS (High Translucent Multi Layered PLUS). KATANA Zirconia HTML PLUS is a pre-sintered zirconia disc 98.5 mm in diameter, which contains a plastic ring. This is designed for all milling systems using this generic-type disc. (Please refer to your milling system’s technical instructions for correct machine operation.) KATANA Zirconia HTML PLUS has 3 available thicknesses: (14 mm, 18 mm and 22 mm) and has 14 shade variations: (A1, A2, A3, A3.5, A4, B1, B2, B3, C1, C2, C3, D2, D3, NW). KATANA Zirconia HTML PLUS consists of 4 graded shade layers. KATANA Zirconia HTML PLUS is recommended for use in fabricating FCZ (Full Contour Zirconia) restorations or the frameworks.

II. Intended Use

KATANA Zirconia is used for the fabrication of the all-ceramic restorations (frameworks, FCZ crowns, FCZ bridges, inlays, onlays and veneers).

III. Sintering program

Temperature	Programing Rate	Holding Time
Room Temp. — 1550°C (2822°F)	10°C/min. (18°F/min.)	—
1550°C (2822°F)	—	2 hrs
1550°C (2822°F) — Room Temp.	-10°C/min. (-18°F/min.)	—

Sintering program 2

Temperature	Programing Rate	Holding Time
Room Temp. — 1400°C (2552°F)	50°C/min. (90°F/min.)	—
1400°C (2552°F) — 1500°C (2732°F)	4°C/min. (7°F/min.)	—
1500°C (2732°F) — 1560°C (2840°F)	10°C/min. (18°F/min.)	—
1560°C (2840°F)	—	16 min
1560°C (2840°F) — Room Temp. (*)	-50°C/min. (-90°F/min.)	—

* The restorations may be removed at 800°C (1472°F) or less depending on the circumstances.

Sintering program 3

Temperature	Programing Rate	Holding Time
Room Temp. — 1450°C (2642°F)	120°C/min. (216°F/min.)	—
1450°C (2642°F) — 1600°C (2912°F)	10°C/min. (18°F/min.)	—
1600°C (2912°F)	—	20 min
1600°C (2912°F) — Room Temp. (*)	-120°C/min. (-216°F/min.)	—

* The restorations may be removed at 800°C (1472°F) or less depending on the circumstances.

IV. Composition

ZrO₂, Y₂O₃, etc.

V. Type and Class (ISO6872:2015)

Type:II/ Classe:5

VI. Physical Properties

Coefficiente de Thermal Expansion (25-500°C (77-932°F)): 10,1×10⁻⁶/K

VII. Directions for Use

(1) Take the disc from the packaging and confirm that the disc does not have a crack or other damage.

(2) Place the disc into the milling machine; then begin the milling process following the milling systems technical instructions.

(3) After milling, remove the restorations from the disc with a diamond bur, etc.

Location & indication	Wall thickness
Anterior crown or bridge	0.4 mm or more
Veneer	0.4 mm or more ¹⁾
Posterior crown or bridge	0.5 mm or more
Inlay or onlay	0.5 mm or more

¹⁾ 0.4 mm or more of this product is for full zirconia veneers. Keep thickness 0.8 mm or more, if it is used for combination with the porcelain.

12. Use the following cross-sectional areas for connectors when fabricating bridges.

Location & indication	Connector cross section	
Anterior	2- or 3-unit bridges	7 mm ² or more
	more than 3-unit bridges	9 mm ² or more
Posterior	2- or 3-unit bridges	9 mm ² or more
	more than 3-unit bridges	9 mm ² or more

- Keep the connected bridge pontics maximum 2 teeth in bridge constructions. When the pontic of 2 teeth continues, keep the connector cross section between the pontic at 12 mm².
- Keep the cantilever bridge to 1 pontic tooth and the connector cross section at 12 mm².
- Choose a shade color that is brighter than the intended color for a thick restoration, as it may look duller depending on the thickness of the restorations.
- When using a sintering furnace for the first time and changing a sintering condition, colors after sintering may vary. Sinter a small piece of Zirconia beforehand and confirm the color.

Caution in conjunction with usage:

- Do not use the disc if there is a crack noticed after removing it from the package.
- Dry milling is recommended. If wet milling is used, the translucency of the disc may be reduced.
- If you find a crack in the restorations, do not use.
- The above sintering recommendation is only a guideline; some adjustments may be required depending on each individual furnace.
- When milling the disc, use caution when approaching the milling bur with the plastic ring to prevent detachment of the disc. Mill the disc as if leaving a zirconia part contacting the internal side of the plastic ring at 2 mm.
- Cut and remove the plastic ring prior to sintering the whole disc.
- Dispose of this product as a medical waste to prevent infection.

Storage:

- Store in a cool and dry place. Keep away from direct sunlight.
- The product should be stored at 10-30°C (50-86°F).
- Do not remove the disc from its packaging during storage.
- The disc is fragile, and requires care when handling.
- The product must be stored in an appropriate place where only dental personnel have access.
- The product must be used by the expiration date indicated on the package.

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

[NOTE]

If a serious accident attributable to this product occurs, report it to the manufacturer’s authorized representative shown below and the regulatory authorities in the country in which the user/patient resides.
“KATANA” and “CERABIEN” are trademarks of NORITAKE CO., LIMITED.

[MD]	Medical Device
-------------	----------------

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