GC Fuji TRIAGE™ CAPSULE

RADIOOPAKE GLASS IONOMER PROTECTION AND TEMPORARY RESTORATIVE MATERIAL IN CAPSULES

For use only by a dental professional in the recommended indications.

- RECOMMENDED INDICATIONS**
- Fluoride protection.
 - Root surface protection.
 - Hyperplastic prevention and control.
 - Protection of the enamel cavity.
 - Temporary filling of endodontic space.
 - Interosseous restoration.
- CONTRAINDICATIONS**
- Pulp capping.
 - Avoid use of this product in patients with known glass ionomer cement hypersensitivity.

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A FISSURE PROTECTION

ROOT SURFACE PROTECTION
HYPERSENSITIVITY PREVENTION AND CONTROL
PROTECTION FOR IMMATURE ENAMEL
 Preparation of the tooth surface (e.g. for fluoride protection or root surface protection)
 a) After cleaning the tooth surfaces prophylaxis with pumice and water / in usual manner, rinse thoroughly with water. Avoid aggravating the operculum.
 If extra retention is desired, application of **CAVITY CONDITIONER** (10 seconds) or **DENTIN CONDITIONER** (20 seconds) is recommended. CAVITY CONDITIONER should be rinsed off thoroughly after application.

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- B. TEMPORARY FILLING OF ENDODONTIC ACCESS**
1. Cleaning the pulp chamber
 2. After appropriate pulp treatment, clean and gently dry the pulp chamber with an air syringe (Fig. B-1).
 3. Fill the chamber with a cotton pellet (Fig. B-2).
 4. See directions in A (above), Section 2.
 5. If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as close as possible to the surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.
 6. See directions in A, Section 3, d, e and f.

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- SHADE**
 Pink, White
- STORAGE**
 For optimal performance, store at temperature of 4-25°C (39.2-77.0°F).
- PACKAGE**
 Single contents per capsule: 0.30 g powder and 0.15 g (0.12 mL) liquid Minimum net volume of mixed cement per capsule: 0.13 mL
- CAUTION**
 1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton swab in alcohol. Flush with water.
 2. In case of contact with eyes, flush immediately with water and seek medical attention.
 3. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
 4. In rare cases the product displays sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

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ЗАЩИТА НА КОРОНЧАТА ПОВЪРХНОСТ
 Preparation of the tooth surface (e.g. for fluoride protection or root surface protection)
 a) After cleaning the tooth surfaces prophylaxis with pumice and water / in usual manner, rinse thoroughly with water. Avoid aggravating the operculum.
 If extra retention is desired, application of **CAVITY CONDITIONER** (10 seconds) or **DENTIN CONDITIONER** (20 seconds) is recommended. CAVITY CONDITIONER should be rinsed off thoroughly after application.

2. Mixin
3. Place
4. Immediately after
5. Immediately remove
6. Immediately after
7. Place
8. Immediately after
9. Immediately after
10. Immediately after
11. Immediately after
12. Immediately after
13. Immediately after
14. Immediately after
15. Immediately after
16. Immediately after
17. Immediately after
18. Immediately after
19. Immediately after
20. Immediately after
21. Immediately after
22. Immediately after
23. Immediately after
24. Immediately after
25. Immediately after
26. Immediately after
27. Immediately after
28. Immediately after
29. Immediately after
30. Immediately after
31. Immediately after
32. Immediately after
33. Immediately after
34. Immediately after
35. Immediately after
36. Immediately after
37. Immediately after
38. Immediately after
39. Immediately after
40. Immediately after
41. Immediately after
42. Immediately after
43. Immediately after
44. Immediately after
45. Immediately after
46. Immediately after
47. Immediately after
48. Immediately after
49. Immediately after
50. Immediately after
51. Immediately after
52. Immediately after
53. Immediately after
54. Immediately after
55. Immediately after
56. Immediately after
57. Immediately after
58. Immediately after
59. Immediately after
60. Immediately after
61. Immediately after
62. Immediately after
63. Immediately after
64. Immediately after
65. Immediately after
66. Immediately after
67. Immediately after
68. Immediately after
69. Immediately after
70. Immediately after
71. Immediately after
72. Immediately after
73. Immediately after
74. Immediately after
75. Immediately after
76. Immediately after
77. Immediately after
78. Immediately after
79. Immediately after
80. Immediately after
81. Immediately after
82. Immediately after
83. Immediately after
84. Immediately after
85. Immediately after
86. Immediately after
87. Immediately after
88. Immediately after
89. Immediately after
90. Immediately after
91. Immediately after
92. Immediately after
93. Immediately after
94. Immediately after
95. Immediately after
96. Immediately after
97. Immediately after
98. Immediately after
99. Immediately after
100. Immediately after

- 1) If a faster set is desired, use a visible light curing device for 20-40 seconds. Place light source as close as possible to the cement surface. This function applies to the Pink Shade. After light cure, the adhesive resin is cured with a curing light.
- 2) In the direction of the nozzle, hold the applicator with the capsule towards you and turn the capsule body.
- 3) To remove
- 4) After placement, when the material starts to lose its glossy appearance (or after curing with the light curing device), apply a finishing coat of GC Fuji COAT LC (light cure) to the sealed area and the margins using a cotton pellet or sponge.
- 5) Finish under air/water spray can be performed 6 minutes from start of mix (chemically set) or 4 minutes if light cure. Use a superfine diamond bur or fine diamond bur with an air syringe (Fig. B-1).
- 6) Apply GC Fuji VARNISH or GC Fuji COAT LC to the surface.

- B. ВРЕМЕННО ОТБУРИВАНЕ НА ЕНДОДОНТИЧЕСКИ КАВИТЕТИ**
1. Cleanse the pulp chamber
 2. After appropriate pulp treatment, clean and gently dry the pulp chamber with an air syringe (Fig. B-1).
 3. Fill the chamber with a cotton pellet (Fig. B-2).
 4. See directions in A (above), Section 2.
 5. If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as close as possible to the surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.
 6. See directions in A, Section 3, d, e and f.

- C. INTERMEDIATE RESTORATIONS**
1. Remove caries and loose debris with hand instruments.
 2. For better retention, it is recommended to gently clean the carious surface with WHITE SHADE or PINK SHADE CONDITIONER for 20 seconds.
 3. After cleaning the tooth surface by blotting with a cotton pellet or gently blowing with an air syringe (Fig. C-1), DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).
 4. Mixin
 5. Placement
 6. See directions in A, Section 3, a and b.
 7. Extrude the mixture directly over the cotton pellet (Fig. B-3).
 8. Polishing (Fig. A-8)
 9. If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as close as possible to the surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.
 10. See directions in A, Section 3, d, e and f.

- D. TEMPORARY FILLING OF ENDODONTIC ACCESS**
1. Cleaning the pulp chamber
 2. After appropriate pulp treatment, clean and gently dry the pulp chamber with an air syringe (Fig. B-1).
 3. Fill the chamber with a cotton pellet (Fig. B-2).
 4. See directions in A (above), Section 2.
 5. If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as close as possible to the surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.
 6. See directions in A, Section 3, d, e and f.

- E. STORAGE**
- For optimal performance, store at temperature of 4-25°C (39.2-77.0°F).
- PACKAGE**
- Single contents per capsule: 0.30 g powder and 0.15 g (0.12 mL) liquid Minimum net volume of mixed cement per capsule: 0.13 mL
- CAUTION**
1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton swab in alcohol. Flush with water.
 2. In case of contact with eyes, flush immediately with water and seek medical attention.
 3. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
 4. In rare cases the product displays sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

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DIRECTIONS FOR USE
PINK SHADE is command set, **WHITE SHADE** is chemical set only

Powder / Liquid Ratio (g / g)	0,30 / 0,15
Mixing Time (sec.)	140*
Working Time (23°C/73°F) (min., sec.)	1'40"
Net Setting Time (23°C/73°F) (min., sec.)	2'30"
Finishing Time (37°C/99°F, from start of mixing) (min., sec.)	6'00"
Finishing Time / Light cure (PINK COMMAND SET) (37°C/99°F, from start of mixing) (min., sec.)	4'00"

A. OCHRANA FISSUR

PREVENCA A KONTROLA HYPERSENZITIVITY
 Preparation of the tooth surface (e.g. for fluoride protection or root surface protection)
 a) After cleaning the tooth surfaces prophylaxis with pumice and water / in usual manner, rinse thoroughly with water. Avoid aggravating the operculum.
 If extra retention is desired, application of **CAVITY CONDITIONER** (10 seconds) or **DENTIN CONDITIONER** (20 seconds) is recommended. CAVITY CONDITIONER should be rinsed off thoroughly after application.

2. Mixin
3. Place
4. Immediately after
5. Immediately remove
6. Immediately after
7. Place
8. Immediately after
9. Immediately after
10. Immediately after
11. Immediately after
12. Immediately after
13. Immediately after
14. Immediately after
15. Immediately after
16. Immediately after
17. Immediately after
18. Immediately after
19. Immediately after
20. Immediately after
21. Immediately after
22. Immediately after
23. Immediately after
24. Immediately after
25. Immediately after
26. Immediately after
27. Immediately after
28. Immediately after
29. Immediately after
30. Immediately after
31. Immediately after
32. Immediately after
33. Immediately after
34. Immediately after
35. Immediately after
36. Immediately after
37. Immediately after
38. Immediately after
39. Immediately after
40. Immediately after
41. Immediately after
42. Immediately after
43. Immediately after
44. Immediately after
45. Immediately after
46. Immediately after
47. Immediately after
48. Immediately after
49. Immediately after
50. Immediately after
51. Immediately after
52. Immediately after
53. Immediately after
54. Immediately after
55. Immediately after
56. Immediately after
57. Immediately after
58. Immediately after
59. Immediately after
60. Immediately after
61. Immediately after
62. Immediately after
63. Immediately after
64. Immediately after
65. Immediately after
66. Immediately after
67. Immediately after
68. Immediately after
69. Immediately after
70. Immediately after
71. Immediately after
72. Immediately after
73. Immediately after
74. Immediately after
75. Immediately after
76. Immediately after
77. Immediately after
78. Immediately after
79. Immediately after
80. Immediately after
81. Immediately after
82. Immediately after
83. Immediately after
84. Immediately after
85. Immediately after
86. Immediately after
87. Immediately after
88. Immediately after
89. Immediately after
90. Immediately after
91. Immediately after
92. Immediately after
93. Immediately after
94. Immediately after
95. Immediately after
96. Immediately after
97. Immediately after
98. Immediately after
99. Immediately after
100. Immediately after

- 1) If a faster set is desired, use a visible light curing device for 20-40 seconds. Place light source as close as possible to the cement surface. This function applies to the Pink Shade. After light cure, the adhesive resin is cured with a curing light.
- 2) In the direction of the nozzle, hold the applicator with the capsule towards you and turn the capsule body.
- 3) To remove
- 4) After placement, when the material starts to lose its glossy appearance (or after curing with the light curing device), apply a finishing coat of GC Fuji COAT LC (light cure) to the sealed area and the margins using a cotton pellet or sponge.
- 5) Finish under air/water spray can be performed 6 minutes from start of mix (chemically set) or 4 minutes if light cure. Use a superfine diamond bur or fine diamond bur with an air syringe (Fig. B-1).
- 6) Apply GC Fuji VARNISH or GC Fuji COAT LC to the surface.

- B. ВРЕМЕННО ОТБУРИВАНЕ НА ЕНДОДОНТИЧЕСКИ КАВИТЕТИ**
1. Cleanse the pulp chamber
 2. After appropriate pulp treatment, clean and gently dry the pulp chamber with an air syringe (Fig. B-1).
 3. Fill the chamber with a cotton pellet (Fig. B-2).
 4. See directions in A (above), Section 2.
 5. If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as close as possible to the surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.
 6. See directions in A, Section 3, d, e and f.

- C. INTERMEDIATE RESTORATIONS**
1. Remove caries and loose debris with hand instruments.
 2. For better retention, it is recommended to gently clean the carious surface with WHITE SHADE or PINK SHADE CONDITIONER for 20 seconds.
 3. After cleaning the tooth surface by blotting with a cotton pellet or gently blowing with an air syringe (Fig. C-1), DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).
 4. Mixin
 5. Placement
 6. See directions in A, Section 3, a and b.
 7. Extrude the mixture directly over the cotton pellet (Fig. B-3).
 8. Polishing (Fig. A-8)
 9. If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as close as possible to the surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.
 10. See directions in A, Section 3, d, e and f.

- D. TEMPORARY FILLING OF ENDODONTIC ACCESS**
1. Cleaning the pulp chamber
 2. After appropriate pulp treatment, clean and gently dry the pulp chamber with an air syringe (Fig. B-1).
 3. Fill the chamber with a cotton pellet (Fig. B-2).
 4. See directions in A (above), Section 2.
 5. If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as close as possible to the surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.
 6. See directions in A, Section 3, d, e and f.

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