Phosphoric Acid Gel Etchant 37.5%

Instructions

1. Apply Kerr Gel Etchant to enamel and dentin surfaces for 15 seconds.
2. Rinse thoroughly for 15 seconds with water. It is important not to splash etch into eyes, onto soft tissues or skin.
3. Dry with clean, oil-free, air without desiccating the dentin. Do not allow etched surface to become contaminated with saliva or blood during the bonding procedure.
4. Proceed with placement of bonding agent.

NOTE: Never use excessive force on syringe plunger to dispense etch which may lead to over dispensing of the material. If resistance is felt replace current tip with a new tip and check flow extraorally.

Refilling of empty 3g intraoral syringe

Caution: For safety reasons, it's mandatory to use gloves, eye protection and mask.

- Unscrew protective cap from 30g Economy syringe, as well as from an empty 3g intraoral syringe.
Screw the connector on Economy syringe on one side and on the empty intraoral syringe on the other side.
- Ensure that the two syringes are perfectly tightened to the connector. **Caution: do not strip the thread**
Fill the empty syringe by slowly and steadily pulling the plunger of the small syringe. Simultaneously, exert light pressure on the plunger of the Economy syringe using the other hand to support the refilling process.
- Avoid overcharging the small syringe.
- Do not point syringes towards people during refilling.
- Do not fill etchant from small syringe back to Economy syringe
- Disconnect the two syringes by an anticlockwise twist
- Replace the screw on cap on the Economy syringe
- Attach the application tip to the small syringe. The syringe is now ready for use.
- To ensure complete product traceabi-

lity, small syringes can be filled only using the product contained in the Economy syringe provided in the same package. Apply on small syringes the two white labels provided in the package, transcribing lot number and expiry date from the Economy syringe.

- In case of any query, please indicate the lot number reported on the Economy syringe or the product envelope
- **NOTE: The two empty syringes included in the package are intended as single use and cannot be filled more than once. Additional empty syringes can be bought ordering the item 34720. Applicator tips are intended as single use as well.**
- **Syringes and applicators cannot be sterilized and must be discarded after the use**

* **Caution:** Contains phosphoric acid. Avoid contact with skin, eyes and soft tissue. In case of contact with skin or eyes, flush immediately with water. Get medical attention for eyes. Do not take internally.

LIMITED WARRANTY

LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

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