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latex gloves may interfere with the setting process of A-Silico-nenes.

– Componds containing eugenol, hemostatic agents or some anaesthetics may also impede setting.

Wörtel

– A-Silicones have a high degree of biological compatibility with no known harmful reactions or secondary effects on patients or dental personnel.

– TempoSIL 2 is designed to set intra-orally.

**Clinical Time**
**Working Time**
≥ 0:50 min
**Setting Time**
≥ 2:00 min
High temperatures accelerate and low temperatures decelerate the setting process.

**Application**
△ Skin secretions, latex gloves and surfaces contaminated by latex gloves may interfere with the setting process of A-Silic-ones. Touch material and the preparation with thoroughly washed and rinsed latex gloves (wash for 15 s with a detergent, rinse with water for another 15 s) or use latex-free gloves.

1. Gently dry the prepared tooth surface by using oil-free com-pressed air. Do not overdry. The best results will be achieved, when the prepared tooth surface looks moist (shiny).

2. Cleanse and dry the inside of the restoration.
3. Extrude the syringe plunger on the syringe and discard.
4. Extrude a small amount of material directly out of the syringe onto a paper towel / tissue until it is evident that base and catalyst are being ejected. This ensures proper mixing.
5. Immediately wipe the syringe orifice to remove excess mate-rial.

6. Attach the Mixing Tip to the syringe by aligning the prongs inside the Mixing Tip with the syringe orifice. Seat onto the syringe and twist clockwise (90°) to lock in place.

7. Apply TempoSIL 2 directly from the syringe into the provision-al. Seat the provisional, applying slight pressure. Secure undisturbed for 30–60 s.
8. After setting, remove the excess material (e.g. scaler).
9. After syringe use, disinfect used Mixing Tip and store the syrin-ge with the used Mixing Tip attached. Remove used Mix-ing Tip prior to using the syringe again.
Bleed syringe to check that material ejects evenly. Attach new Mixing Tip.

**Shelf life and storage**
TempoSIL 2 must be stored at temperatures from 15 °C - 23 °C / 59 °F - 73 °F. Avoid exposure to direct sunlight or other heat sources.

Marking

The expiry date and **LOTTI**-number are marked on containers and outerpack.

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