## Depural™ Neo

**Cleaning And Polishing Paste**

**Characteristics:**
DEPURAL NEO is a cleaning and polishing paste without fluorine.

**Indications:**
Slightly abrasive DEPURAL NEO paste is intended for professional tooth cleaning in a dental surgery. It is used for final cleaning and polishing of tooth surface after prophylactic as well as therapeutic removal of dental tartar, for removal of dental plaque and pigmentations prior to preservative tooth treatment or prior to surgical or controlled tooth whitening.

**Contraindications:**
DEPURAL NEO paste must not be used in case of known allergy to any of the product constituents. The product is not intended for routine oral hygiene.

**Instructions for use:**
Express necessary amount of the cleaning paste from the tube onto a suitable plate and pick up this expressed paste into a rotary brush or a prophylactic rubber cup. Clean the wet tooth surface using a slight pressure and low revolutions, change the direction of rotation as necessary. Adequate quantity of the cleaning paste provides sufficient cooling of the rotary instrument. Washing-away of the paste during the treatment is assured by means of a suction apparatus.

**Warning:**
DEPURAL NEO contains abrasives and flavouring agents which could cause irritation in case of accidental contact with the eye. In case of contact with eyes, rinse with plenty of running water, and if the complaints persist, seek professional medical advice.

**Caution:**
Wear protective devices (goggles) during the work. Use a towel to protect the patient from staining.

**Waste disposal:**
Empty packages and unused residues of the material should be disposed of together with common waste materials produced in the workplace.

**Storage:**
Store in a dry and dark place at 5 °C to 25 °C, in a well sealed container.

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