DURACRYL® PLUS

CS

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Self-curing base resin

Characteristic:

The chemically initiated methacrylate resin having better colourfastness. The new shades of DURACRYL® PLUS correspond with the basal resin SUPERACRYL® PLUS.

Indication:

- For restoration of removable denture.
- For adding the teeth into prostheses.
- For preparing the immediate prostheses.
- For the indirect rebasings.

Contra - indications: Not described.

Undesirable effects: The potential sensitivity on methylmethacrylate, especially with the allergic people.

Preparation: DURACRYL® PLUS is mixed in the volumetric ratio 3 : 1 (three parts of powder, 1 part of liquid). It is recommended to pour consecutively the powder into the liquid. The mixture is to be thoroughly mixed up. The improvement of the physical material properties may be reached by the polymerization in the simple pressure vessel with water 30 - 40°C

warm, under the pressure of 0,2 - 0,3 MPa. The polymerization having been finished, the resin may be immediately worked up.

Warning:

Powder: H317 May cause an allergic skin reaction. H400 Very toxic to aquatic life.

P262 Do not get in eyes, on skin, or on clothing. P271 Use only outdoors or in a well-ventilated area. P273 Avoid release to the environment.

Liquid: H225 Highly flammable liquid and vapour. H315 Causes skin irritation. H335 May cause respiratory irritation. H317 May cause an allergic skin reaction. H302 Harmful if swallowed. H312 Harmful in contact with skin. H332 Harmful if inhaled.

P210 Keep away from heat/sparks/open flames/hot surfaces. – No smoking. P262 Do not get in eyes, on skin, or on clothing. P270 Do not eat, drink or smoke when using this product.

Waste material disposal: The residue of unused product and contaminated packaging material should be transferred to a company authorized to dispose of waste materials. Observe your local legal regulations.

Storage: Store in a dry and dark place at 5 °C to 25 °C, in a well sealed container. Store the package in a well ventilated area, away from heat sources. Smoking is prohibited.

Shelf life: 4 years

Packing:

powder: 500 g - in rosa shades U, V, Z and colourless in the shade O.

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