

Adper™ Prompt™ L-Pop™ Self-Etch Adhesive

Preparation:

- Thoroughly clean teeth to remove plaque and debris from enamel surfaces and fissures. Rinse thoroughly with water and dry. Note: Do not use any cleaning medium that contains oils or fluoride.
- Prepare adhesive according to dispensing instructions (see opposite side).

- Brush adhesive onto surface to be sealed; massage for 15 seconds applying moderate pressure. **Do not shorten this time. Proper bonding will not occur if the solution is simply applied and dispersed.**



- Use a gentle stream of air to **thoroughly dry** the adhesive to a thin film. A second layer is not required for bonding sealants.



- Apply a **light-cured** sealant. (Note: Adper Prompt self-etch adhesive is not indicated for self-cure sealants)



- Light-cure the sealant and adhesive together with a halogen, LED or plasma light for the amount of time specified to cure the sealant.



Adper™ Prompt™ L-Pop™ Self-Etch Adhesive



Dispensing:

- Squeeze the material from the red reservoir into the yellow (middle) reservoir.
- The middle reservoir will expand indicating proper transfer of fluid.



- Carefully fold back the red reservoir until it meets the flat face of the yellow reservoir.
- **Keep the red reservoir tightly squeezed while bending the package to prevent the material from flowing back.**



- Squeeze the liquid from the yellow reservoir into the green reservoir.



- Apply a churning or spinning motion to the applicator for 5 seconds to mix the adhesive and fully coat the applicator tip.
- **A lightly yellow-tinted applicator tip indicates proper activation.**

2 of 2

3M ESPE Technical Hotline 1-800-634-2249
Please refer to instructions for more detailed information
as well as precautionary and warranty information.

3M, ESPE, Adper, Prompt, and L-Pop are trademarks of 3M or 3M ESPE AG. Used under license in Canada.

70-2009-3551-1
44-0007-3459-8
© 3M 2003