Calibra® Universal

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3.3 Cementation technique

Self-cure		Dual Cure - Non-Light Transmissible Restorations	Light Cure - Light Transmissible Restorations
Type of Restoration	All	PFM, Zirconia, Alumina, opaque ceramics & composites	Translucent ceramics & composites
1. Fill and seat crown	Up to 2 min	Up to 2 min	Up to 2 min
	Protect restoration from movement during gel phase cleanup through final set		
2. Remove Excess			
Gel stage from seating: 1-2 min	Up to 5 sec per surface/ light cure	Up to 5 sec per surface/ light cure	
Gel Duration: 1 min	45 sec	45 sec	
3. Stabilize	Constant Occlusal Pressure	Light cure margins 20-40 sec	Light cure margins 20-40 sec
4. Final Set	6 min from start of mix of mix	6 min from start of mix of mix	Upon completion of light curing each surface ¹⁰ sec
1 Buccal and Lingual ²	Buccal, Lingual and Oclusal		

Danger of injury due to excessive force
 • Apply slow and steady pressure on the syringe
 • Do not use excessive force - syringe rupture may result

3.3 Dual Barreled syringe dispensed

1. Remove syringe cap. Dispense and discard a small amount of material from the dual-barreled syringe.

2. Hold the syringe firmly from both ports. Holding syringe vertically, carefully wipe away excess soot and contamination from the cross contaminate and cause obstruction of the ports. Save syringe cap for replacement following use.

3. Install a mixing tip on the cartridge by lining up the v-shaped notch on the outside of the mixing tip with the v-notch snap on the syringe flange. Turn colored mixing tip cap 90 degrees in a clockwise direction to lock in place on syringe flange.

4. Gently push the plunger to begin the flow of material. DO NOT USE EXCESSIVE FORCE. If force is encountered, remove syringe from operating field, remove and discard mixing tip. Check for obstruction and confirm material is flowing from both syringe barrels. Wipe barrels and install new mixing tip as outlined above. Dispense a small amount through the mixing tip onto a mixing pad and discard.

5. Wait 10 seconds, then repeat steps 3-4. Verify complete seating. A gentle rocking or vibratory motion may be helpful to increase optimal sealing.

6. Protect restoration from contamination until the final set of the cement (6 minutes from start of mix or in the case of light-transmissible restorations, completion of light curing).

3.4 Cleaning Marginal Excess Calibra Universal Cement

3.4.1 Self-cure cleanup

The excess cement will reach the "gelled" state after approximately 1-2 minutes in the mouth, allowing easy removal. Excess cement will remain in the "gelled" state for approximately 1 minute. If exposed to direct operatory light, "gel" state may reach sooner and remain "gelled" for a shorter period. Immediately after reaching the "gelled" state, it is recommended to use a dental curing light (e.g., 700nm) to remove excess cement. Complete excess removal using an instrument such as a rubber-dam or scalpel or an explorer. **NOTE:** Cement within the crown has not yet set. Do not move, torque, or disturb the crown during cleanup. **Technique Tip:** Following all excess removal, exposed margins may be light cured, cured or sealed to assist restoration stabilization.

3.4.2 Optional dual cure cleanup

Due to the dual cure property of Calibra Universal Cement, the operator has the option of utilizing a curing light to facilitate cleanup. Light curing to facilitate cleanup must be accomplished within the first minute following intraoperative insertion. Excess cement cleanup may begin immediately following a brief exposure with the curing light. Conventionally prepared gasketed or occlusal or LED lights producing only one peak wavelength of 470nm are recommended. Use of a curing light with a broad spectrum of wavelengths or power, a broad spectrum output between 400-600 nm, may produce unexpected results. Light curing cements after one minute, or continuously at one spot, for more than 5 seconds, at any time, will cause cement to adhesive setting, making cleanup difficult.

Immediately after gassing is verified, briefly light cure excess cement at the margins by constantly moving the curing light tip around the margins for no more than 5 seconds per surface. Excess cement will reach a "gelled" state after this brief (<5 second) cure.

Residual excess cement – overcure during cleanup:

• Monowave 400nm LED curing lights are recommended

• Dual or broad spectrum lights may cause premature hardening of excess cement

• Evaluate curing light/cement interaction in laboratory prior to clinical use

The excess cement will remain in the "gelled" state for approximately 45 seconds following light exposure. All excess cement must be removed before final self-cure set is achieved, as outlined above. **NOTE:** Cement within the crown has not yet set or gelled. Do not move, torque, or disturb the crown during cleanup. Following all excess removal, exposed margins may be light cured 20-40 seconds to assist restoration stabilization.

Technique Tips:

• When simultaneous cementing multiple single units or bridgework, it is recommended to employ the light cure cleanup on one or two adjacent units only, allowing other units' excess to self-cure, providing ample cleanup time.

• Ensure all excess cement has been removed from gingival sulcus and/or undercuts before final curing. Material is adhesive when set and will be difficult to remove.

• Clean excess cement from metal instruments immediately as set cement will adhere to the instrument.

3.5 Curing

• Non-Light-Transmissible restorations (Self-Cure)

For metallic, thick or heavily opacified ceramic or composite, or restorations that otherwise impede the transmission of light, once cleanup is completed and restoration is stabilized, allow Calibra Universal cement to polymerize on its own. Once cleanup is completed and restoration is stabilized, light all areas of the restoration with a visible light, curing unit designed to cure CQ initiated methacrylates (spectral output including 470nm), with a minimum output of 550mW/cm² for 10 seconds from each direction – buccal, lingual and oclusal. Following the light curing, check and adjust occlusion and polish as necessary. Material may then be dismissed.

Inadequate retention due to insufficient stabilization

• Self-cure restoration during cleanup

• Protect restoration from movement during setting

• Allow cement to self-cure without disturbing for 6 minutes

• Light-Transmissible restorations (Light-Cure)

For translucent, translucent/opaque, or composite restorations, Calibra Universal Cement may be visible light cured. Once cleanup is completed and restoration is stabilized, light all areas of the restoration with a visible light, curing unit designed to cure CQ initiated methacrylates (spectral output including 470nm), with a minimum output of 550mW/cm² for 10 seconds from each direction – buccal, lingual and oclusal. Following the light curing, check and adjust occlusion and polish as necessary.

Inadequate polymerization due to insufficient curing

• Check compatibility of curing light

• Check curing cycle

• Check curing output before each procedure

3.6 Finishing and polishing

• Removal of resin flash is best accomplished with the Enhance® Finishing System of points, cups and discs. The Enhance System will remove flash and finish restoration margins without removal of or trauma to the enamel.

• Polishing restoration using Enhance® PoGo® Polishing system or Prisma® Gloss™ Polishing Paste and Prisma Gloss Extra Fine Polishing Pastes. (See complete Directions for Use supplied with polishing product chosen).

• All shades of Calibra Universal Cement are radio-opaque, with a 1 mm radio-opacity equivalent to 1.8 mm radio-opacity of aluminum. Aluminum has a radio-opacity equivalent to that of dentin. Thus 1 mm of material having a radio-opacity equivalent to that of dentin.

4. HYGIENE

Cross-contamination

Infection control: Do not reuse single use products. Dispose of in accordance with local regulations.

• To prevent the syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringes are handled with clean/disinfected gloves. Do not reuse syringes if contaminated.

• Syringe cannot be reprocessed. Dispose of contaminated syringe in accordance with local regulations.

To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands, or oral tissues, use of a protective barrier is recommended. The use of protective barriers is an additional precautionary measure against gross debris but not against all contamination.

Incidental contact of the syringe with water, soap or a water-based hospital-level disinfection solution will not damage syringe body. Do not allow any solution contact with contained material. Discard composite material that has been in contact with any fluid or non-sterile instrument.

NOTE: Destruction of the label by vigorous wiping. Wipe syringe gently.

5. LOT NUMBER AND EXPIRATION DATE

1. Do not use after expiration date. ISO standard uses: "YYYY-MM-DD."

2. The following numbers should be quoted in all correspondences:

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