DE Lunos® Polierpaste Two in One
Neutral / Orange / Minze

EN Lunos® Polishing Paste Two in One
Neutral / Orange / Mint

FR Lunos® Pâte de polissage Two in One
Neutre / Orange / Menthe

IT Lunos® Pasta lucidante Two in One
Neutro / Arancia / Menta

ES Lunos® Pasta pulidora Two in One
Neutral / Naranja / Menta

DE Produktinformation

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EN Product information

Lunos® Polishing Paste Two in One Neutral / Orange / Mint

„Two in One Prophy Paste“ is available in the following variants:

- Neutral (REF: CPZ710A1901)**, without fluoride or flavouring agents
- Orange (REF: CPZ720A1901)**, without fluoride, contains limonene
- Mint (REF: CPZ730A1901)**, contains 679 ppm total fluoride (sodium fluoride) and peppermint oil

Product description

The „Two in One Prophy Paste“ is a medical device. This product is only intended for professional dental applications and must only be used by trained specialists.

„Two in One Prophy Paste“ is free of animal ingredients, gluten, lactose, fructose and microplastic.

Indication

- its use is permissible only together with polishing instruments suitable for the prophylactic purpose
- for gentle removal of discolouration and plaque as well as subsequent polishing of supragingival tooth surfaces in a single process
- for preparation and cleaning of tooth surfaces prior to application of fluoride products, application of bleaching products and insertion of permanent prosthetics such as inlays, onlays, crowns, veneers, etc.
- suitable for cleaning prior to application of orthodontic brackets and use of composite restoration materials. Here, the fluoride-free variants „Orange“ or „Neutral“ are to be used preferably.
- works on the innovative principle of „self-reduction“. In other words, the initially larger rod-shaped polishing particles are reduced in size and broken down during the polishing process. As a result, in a single process first the plaque is removed quickly and thoroughly in the first 10 seconds of cleaning, and immediately afterwards the supragingival tooth surfaces are gently smoothed and polished.

„Two in One Prophy Paste“ has an RDA of < 28* and is suitable for children and adults provided a trained swallowing reflex is ensured.

Contraindications

„Two in One Prophy Paste“s are contraindicated in patients with known allergies to ingredients of the pastes (e.g. limonene; peppermint oil). If the patient has known allergies to limonene or peppermint oil, the „Neutral“ variant of „Two in One Prophy Paste“ is to be recommended.

Warning

Do not swallow. Avoid contact with the eyes. In case of contact with the eyes, rinse with plenty of water. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

Precautions

Use suitable protective equipment when working with „Two in One Prophy Paste“: patients and operators need to wear suitable eye protection, e.g. protective goggles. The operator needs to wear suitable protective equipment in the form of mouth and nose protection and protective

gloves.

„Two in One Prophy Paste“ may only be used by appropriately trained dental specialists.

The „Mint“ variant with fluoride does not replace a dedicated fluoride therapy if this is indicated. On the day of treatment with the „Mint“ variant, do not use any other fluoride-containing products.

Prior to using fluorescence-based caries diagnosis equipment, the „Neutral“ or „Orange“ variants should be used, not the „Mint“ variant.

Clean the surfaces of the dental unit with a moist cloth following contact with the Prophy Paste.

Preparation

For first application, the tamper-evident seal of the tube is to be removed. Squeeze out a sufficient amount of Prophy Paste into an application ring, e.g. Lunos® Prophylaxis Ring, dappen glass or similar suitable vessel. Take up Prophy Paste with a suitable polishing instrument and begin the polishing process. Please note that the self-reducing effect of the paste starts after approx. 10 seconds. Keep polishing until the polishing particles have broken down into fine particles, so that a smoothing effect can be achieved once the plaque has been removed.

Please note that sufficient Prophy Paste must be present between polishing instrument and tooth, in order to avoid generating heat on the tooth. For this reason, please also take care not to exceed the maximum speed of rotation recommended by the polishing instrument manufacturer. Please observe the directions for use of the device manufacturer.

Storage and shelf life

Please make sure that the tube is carefully closed again immediately after removal of paste, to prevent the paste from drying out.

Store in enclosed, well-ventilated rooms away from moisture and bright light between 0°C and 25°C. Keep out of direct sunlight. When the storage conditions are observed, the shelf life is 36 months, or 12 months after opening.

Disposal

Dispose of in accordance with the local official regulations.

Composition

Two in One Neutral: Silicates, glycerin, distilled water, hydroxyapatite, acidity regulator, preservatives (phenoxyethanol, parabens), thickening agent, emulsifying agent, colouring agents (C.I. 77891)

Two in One Orange: Silicates, glycerin, distilled water, hydroxyapatite, acidity regulator, preservatives (phenoxyethanol, parabens), thickening agent, emulsifying agent, colouring agents (C.I. 47005, C.I. 77891), orange flavouring (contains limonene)

Two in One Mint: Silicates, glycerin, distilled water, hydroxyapatite, acidity regulator, preservatives (phenoxyethanol, parabens), thickening agent, emulsifying agent, sodium fluoride (1500 ppm = 679 ppm F-), colouring agents (C.I. 42090, C.I. 47005, C.I. 77891), peppermint oil

* Method according to Stookey/Schemehorn with reference substance pumice stone for prophylaxis pastes [J Dent Res, 1979 58(2): p.588-592], relative RDA value in comparison to product „C“ = RDA 27
Measurement results: Product „C“ = 255 / Two in One = 260

DÜRR DENTAL (Products) UK Ltd, 14 Linnell Way – Telford Way Industrial Estate Kettering, Northants NN16 8PS, United Kingdom ☎ +44 1536526740

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DÜRR DENTAL ITALIA S.r.l. Via Dell'Artigianato n. 8, 20053 Muggiò (Mi), Italy ☎ +39 0395970300
urochemie GmbH + Co. KG