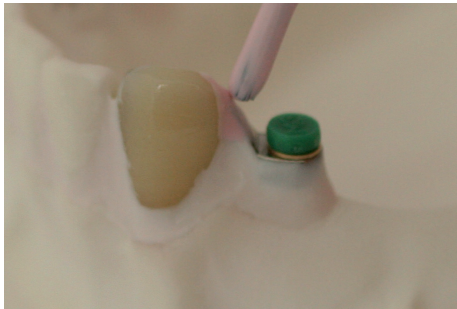
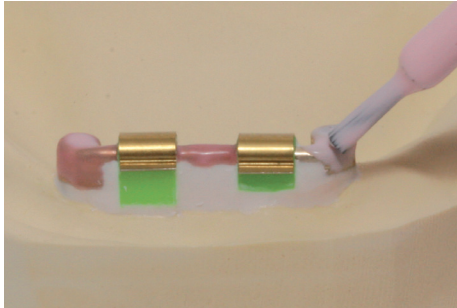
## PRECI-SEP

# 6501 15 ML

### AXIAL

#### ATTACHMENTS

CEKA AXIAL  
PRECI-CLIX AXIAL  
PRECI-CLIX RADICULAR  
PRECI-BALL

### EXTRACORONAL ATTACHMENTS

CEKA EXTRACORONAL  
PRECI-VERTIX  
PRECI-CLIX EXTRACORONAL  
PRECI-SAGIX  
PRECI-S2

### PROFILE

#### ATTACHMENTS

PRECI-BAR  
PRECI-HORIX  
PRECI-CLIP

### DIGITAL

#### LIBRARIES

### SPECIALTIES

PRECI-PROFILE  
PRECI-POST  
CEKA SOL  
CEKA SITE  
CEKA BOND  
3C-BOND  
CEKA Multi - CEKA Multi O  
PERMA-RET  
**PRECI-SEP**  
PLASTICWAX  
EXPANDO  
MEASURING GAUGE

#### SIDE EFFECTS, WARNING AND PRECAUTIONS

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat products containing titanium.  
Do not use products containing nickel in case of nickel allergy.  
The accessories RE H79 and H 35 must be used outside the mouth.

Prospectus no 294.M1-ed.H  
© Copyright ALPHADENT NV 03/2019  
BTW BE 0414.789.321 RPR Kortrijk

CEKA  
PRECI-LINE

WWW.CKPL.EU

ALPHADENT NV  
Mannebeekstraat 33,  
8790 Waregem, Belgium  
T +32 56 629 531  
E info@ckpl.eu

CEKA  
PRECI-LINE

## SPECIALTIES

### PRECI-SEP

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WWW.CKPL.EU

# PRECI-SEP

PRECI-SEP is a latex-based separator, spacer and release agent for multiple dental applications:

- use as a spacer for precision attachments
- use as a spacer for composite and acrylic provisional restorations
- use as a spacer for customized abutments when modelling with self-cure or light-cure acrylic
- protects against sandblasting and acids
- use as a spacer with vacuum formed trays
- release agent for crowns when pouring a pick-up impression

Apply with a brush and let dry. To speed up drying, use compressed air. The pink colour will become translucent. Use multiple layers when more thickness is desired. Peels off clean.

Use a few drops of distilled water if PRECI-SEP thickens.

**CAUTION:** contains natural rubber and may cause allergic reactions.

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