**VOCO**

# Provicol® QM Plus

In accordance with DIN EN ISO 3107

## Instructions for use

### Product description:

**Provicol QM Plus** is an easy-to-use eugenol-free temporary luting cement, which does not affect the setting of composite-based restoratives and luting materials. The calcium hydroxide contained in **Provicol QM Plus** helps maintain tooth vitality.

Working time: ≥ 1 min (23 °C)  
Setting time: ≤ 4 min (37 °C)

### Indications:

- Temporary luting of provisional restorations (crowns, bridges, inlays and onlays)
- Temporary luting of definitive restorations (crowns, bridges, inlays and onlays)
- Temporary obturation of small single-surface cavities

### Method of use:

#### 1. Preparation:

Ensure that surfaces are clean before cementation begins (see supplementary advice below). Remove the protective cap from the QuickMix syringe. Due to technical reasons, there may be slight differences in the filling levels of the syringe chambers prior to first use. Therefore remove the syringe cap and extract material until both chambers deliver uniformly. Then fit a mixing tip type 10. The notches on the syringe and the mixing tip must line up exactly. Turn the mixing tip 90° clockwise to lock the tip into place. The warranted product characteristics can only be achieved when using the original Sulzer Mixpac mixing tips as approved by **VOCO**. All the mixing tips for use with our respective products are available from **VOCO**.

#### 2. Placing the restoration/application:

Apply **Provicol QM Plus** to the dry temporary restoration and/or the prepared tooth surface. Then place the restoration by applying gentle pressure, generating excess material in the process. The working time at room temperature is at least 1 min. Leave the used mixing tip on the syringe to seal it until the next application. Alternatively, re-seal the syringe with the protective cap after application.

#### 3. Curing:

- Any excess material can be removed using the following methods:
- Remove excess amounts of not fully cured **Provicol QM Plus** using a foam pellet or similar.
- Wait for material to cure fully (observation of progress in the mouth) and then remove excess cured cement using a suitable instrument.

Contact points and interproximal areas need to be cleaned with dental floss. The sulcus must be free of material residues, as they may irritate the gingiva.

Curing is completed 4 minutes after insertion. The restoration must be held in position during the polymerisation phase.

#### 4. Removal:

Use a suitable instrument to remove the restoration and completely remove all **Provicol QM Plus** residues before final luting. This can be done with a suitable instrument (curette, scaler, probe) and, if required, with polishing paste.

### Information, precautionary measures:

**Provicol QM Plus** contains modified colophony. **Provicol QM Plus** should not be used in case of known hypersensitivities (allergies) to this ingredient.

In case of contact with the eyes, immediately rinse with plenty of water and seek medical advice.

The sensitivity of cores can be reduced by using desensitising preparations prior to temporary luting.

Appropriate measures for pulp protection are recommended in cases of exposed pulp or insufficient residual dentine thickness. A harder cement is the better choice for multi-surface cavities and those which have an extensive surface area, as well as long-term temporary obturations.

**Provicol QM Plus** residues are easily cleaned off restorations after removal if they are briefly treated with cooling spray outside the mouth.

### Supplementary advice:

Wetting prepared teeth with a little water before cementation makes subsequent removal of the restoration easier.

### Storage:

Store at temperatures of 4 °C to 23 °C.

Do not use after expiry date.

Allow **Provicol QM Plus** to reach room temperature before use!

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA1098 E1 1117 99 © by VOCO
**VOCO GmbH**