VOCO

Registrado® Scan

GB

Instructions for use

**DE**

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FR

Mode d'emploi

Product description:

Registrado Scan is a fast-setting, addition-curing silicone impression material for bite registration. It is characterised by a viscous consistency, great dimensional stability and a high final hardness.

Registrado Scan can be used for the optical compilation of data for CAD/CAM/CIM technology without powdering.

Indications:

- Occlusal bite registration
- Fixing of face bow registrations
- Location material for intraoral pin tracing templates

Application:

1. Preparation:

To dispense **Registrado Scan**, use a **VOCO** Dispenser type 2 or any other suitable dispenser.

Due to technical reasons there may be slight differences in the filling levels of the two cartridge chambers. Therefore, remove cartridge cap after placing the cartridge into the dispenser type 2 and extract material until it is delivered uniformly from both chambers.

Fit a mixing tip type 16 and extract material by pressing the lever. The material is completely and automatically mixed in the correct ratio in the mixing tip.

An intraoral tip type 2 may be placed on the mixing tip type 16 to facilitate finer, precise application.

2. Handling:

In case of occlusal bite registration, apply **Registrado Scan** directly onto the upper or lower dental arch of the patient.

The patient is requested to bring his/her teeth into the habitual occlusion position without functional movements and hold the position for at least 45 s. **Registrado Scan** can then be removed from his/her mouth.

Working time (at 23 °C)	max. 30 s
Time in mouth (at 37 °C)	min. 45 s

3. Post-processing:

Disinfection with an aqueous disinfectant is recommended prior to further processing of the occlusal bite registration produced. When doing so, observe the instructions for use of the disinfectant used. Dry the registration material before the scanning procedure. After setting, **Registrado Scan** can be processed in a simple and controlled manner using a scalpel, scissors or a bur and remains permanently dimension stable.

Information, precautionary measures:

Registrado Scan contains platinum catalyst. **Registrado Scan** should not be used in case of known hypersensitivities (allergies) to this ingredient.

Do not use **Registrado Scan** in combination with condensation-curing silicones. The setting behaviour of **Registrado Scan** can be affected by the use of latex gloves, hand creams, cleaning agents, mouth rinses, retraction solutions and residues of other materials (e.g., impression materials). Consequently, these must be removed prior to application (rinsing off with spray if necessary). Incompletely cured silicone material should not be allowed to come into contact with clothing.

The warranted product characteristics can only be achieved when using the original Sulzer Mixpac mixing tips as approved by **VOCO**. All the mixing tips for use with our respective products are available from **VOCO**.

Storage:

Store at temperatures of 4 °C to 23 °C. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VOCO GmbH