SimpliShade™

Bulk Fill

Nanohybrid Restorative Composite

Instructions for Use

DMC4362E_Rev.03
2024-01

Kerr Corporation
1889 W. Mission Blvd.
Pomona, CA 91766 USA
1-800-KERR-123
kerrdental.com

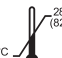
Rx only
*for US law only

Kerr™

SYMBOL DESCRIPTION

Full explanation of symbols used on Kerr packaging is located at:

<http://www.kerrdental.com/symbols-glossary>

	Catalogue number
	Manufacturer
	Batch Code
 www.kerrdental.com/IFU	Consult Instructions for Use or Consult electronic instructions for Use at www.kerrdental.com/IFU
	Caution
Rx only	Caution: Federal law (USA) restricts this device to sale by or on the order of a licensed healthcare practitioner.
Rx only *for US law only	
	Use-by date
	Do not re-use
	Contains nano materials
	Keep away from sunlight
	Temperature storage limitations: low 2°C/36°F and high 28°C/82°F

CONTENTS

	1. Unidose™ Tip
	2. Syringe

en - ENGLISH

SimpliShade™ Bulk Fill

Nanohybrid Restorative Composite

ISO 4049: Polymer-Based Dental Restorative Material, Class 2/Group 1

GENERAL INFORMATION

SimpliShade™ Bulk Fill is an intra-oral, light-cured, esthetic, resin-based nanohybrid dental restorative that is designed for the placement of direct restorations. SimpliShade Bulk Fill is designed to be used after placement of a dental adhesive, such as a product from the Kerr OptiBond™ family.

SimpliShade Bulk Fill is available in Unidose™ compules and syringe delivery systems that are suitable for occlusal surfaces. The product allows a technique in which a cavity up to 5 mm in depth can be filled and cured in a single increment.

SimpliShade Bulk Fill is offered in one universal shade to optically blend with a range of VITA™ shade guide equivalents.

This composite can be polished with a wide variety of polishing discs, rubber cups, and brushes.

PRODUCT COMPOSITION

SimpliShade Bulk Fill composite is radiopaque with about 250% aluminum (100% aluminum has a radio-opacity equivalent to that of dentine and 200% aluminum is equivalent to enamel).

The fillers are comprised of chemically infused mixed oxides, pre-polymerized filler, barium glass filler, silica, and ytterbium trifluoride (total filler loading of about 82.5% by weight or 66% by volume). The inorganic filler content is up to 75% by weight or 55% by volume. SimpliShade Bulk Fill has a primary particle size range of 40 nm to 0.4 µm. The product contains Bis-EMA, Bis-GMA, UDMA, and TEGDMA resins.

None of the ingredients in the product contain pharmacologically active components.

INDICATIONS FOR USE

SimpliShade Bulk Fill is indicated for direct placement of anterior and posterior restorations, core build-ups, and repair of defects in restorations.

CONTRAINDICATIONS

None known.

WARNINGS

- Do not use this product on patients with known allergies to methacrylate and/or other device components.
- Avoid product application in close proximity to the pulp tissue since exposure to this product may injure the pulp.
- Avoid skin, eye, and soft tissue exposure to this product, since uncured methacrylate may cause an irritation reaction. Wash thoroughly with water after contact.
- In case of contact with eyes, rinse immediately with water and seek medical attention.
- Do not take internally.

PRECAUTIONS

- This product is intended to be used by qualified dental professionals on a general patient population.
- Provide the patient with protective eyewear.
- Use a rubber dam to isolate the oral cavity.
- Avoid contamination of the preparation with fluids during restorative placement.
- Use of protective gloves and a no-touch clinical technique is strongly recommended.
- If the composite is touched with commonly used gloves, acrylate may penetrate the gloves. Dispose of gloves immediately and wash hands.
- Do not use the restorative material on surfaces that are contaminated with hydrogen peroxide (H₂O₂) or eugenol, as its residues can adversely affect the polymerization of the composite.
- Protect the material from ambient light that may initiate polymerization.
- Do not open the Unidose composite foil package until initial use. Use the product within 6 months of opening the pouch, but always ensure that the composite is not past the expiration date that is printed on the label.
- The Unidose tips are designed for SINGLE PATIENT USE ONLY to prevent cross-contamination between patients.

ADVERSE EVENTS

If a serious incident occurs with this medical device, report it to the manufacturer and to the competent authority for that country in which the user and/or patient is established.

STEP-BY-STEP INSTRUCTIONS

Wear personal protective equipment when using and handling this product.

Do not open the Unidose composite foil package until initial use. Use the product within 6 months of opening the pouch.

- Isolate the tooth with your preferred isolation method.
- Prepare the tooth using your preferred rotary instruments and using contemporary preparation designs. SimpliShade Bulk Fill composite offers excellent blending (chameleon effect) which may be further enhanced by the use of small bevels on the preparation margins. If replacing a restoration, be sure to remove all existing restorative material from the operative site.
- If a pulp exposure has occurred, place a small amount of calcium hydroxide, followed by the placement of a conventional glass ionomer base.
- SimpliShade Bulk Fill is compatible with all methacrylate dental adhesive systems. Complete the bonding procedure per manufacturer's instructions.
- If desired, place a liner using a low viscosity flowable composite and light cure according to the manufacturer's instructions.
- In situations of discoloration of the tooth structure, a universal opaquer composite shade can be used to block out the discoloration or hide a metal portion if it exists.
- SimpliShade Bulk Fill features optimal, non-sticky handling. It permits the use of hand instruments to define most of the restoration anatomy, which will help minimize the time and effort that are associated with rotary instrumentation.
For Unidose delivery: Place the Unidose tip into a dispensing gun and remove the Unidose cap. Inject composite material from the Unidose tip directly into the cavity, starting with the deepest portion.
For syringe delivery: Extrude the material from the syringe prior to immediate use. After extruding material onto a pad, back off the syringe at least 2 full turns, to relieve pressure on the material.
- For Class I, Class II, and core-build-ups, apply the composite up to a 5 mm* increment

taking care to properly adapt the material to the cavity surface.

- Light cure the composite*. Refer to the table at the end of these instructions for recommended curing times at various light intensities.
- Adjust occlusion as needed. Complete the finishing and polishing using your preferred polishing system.

*RECOMMENDED CURING TIMES

Adequate curing is essential for restorative function. Uncured or undercured composite material is far more likely to experience some form of clinical failure. Kerr recommends regular inspections of your curing lights to ensure they are functioning and providing the expected irradiance. SimpliShade Bulk Fill is intended to be cured by a Halogen light with a minimum light intensity of 650 mW/cm², or an LED light with a minimum light intensity of 1,000 mW/cm² and a light wavelength output within the 400-520 nm range. Please refer to the table below for curing recommendations. All times refer to a single occlusal surface cure. In any posterior restoration, Kerr recommends additional 10 second cures on both the buccal and lingual/palatal surfaces of the tooth.

For all other lights, see the manufacturer's recommendations.

Curing Method	Universal Shade
Incremental depth	Up to 5 mm
Optilux™ (or light with output 650-1,000 mW/cm ²)	20 seconds
Demi™ Ultra/Demi™ Plus (or light with output > 1,000 mW/cm ²)	10 seconds

CLEANING AND DISINFECTION

Unidose Tips: Do not re-use Unidose tips. Dispose of Unidose tips in accordance with state and federal laws that are applicable to that country.

Unidose dispensing guns: Refer to the reprocessing instructions for the manufacturer of the tip gun you are using.

Composite syringes: To prevent syringes from getting contaminated with saliva or other bodily fluids, use protective barriers. Always handle the syringe with clean/disinfected gloves.

Always replace the syringe cap after each use to prevent the resin from hardening in the syringe.

Kerr recommends the use of an intermediate level disinfectant, such as CaviCide™ and CaviWipes™. Refer to the manufacturer's instructions for recommended contact time, dilution, and solution temperature.

SHELF LIFE, STORAGE AND DISPOSAL

SimpliShade Bulk Fill should be stored at ambient temperature between 2°C to 28°C (36°F to 82°F). Avoid excess heat. If the product is stored in a refrigerator, allow the product to reach room temperature prior to use.

The expiration date is marked on the packaging. Do not use the restorative after the expiration date. Once the foil pouch is opened, use within 6 months and prior to the expiration date that is printed on the label.

For proper disposal, always follow federal or state laws.

LIMITED WARRANTY - LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's products. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

Kerr, SimpliShade, OptiBond, Unidose, CaviCide and CaviWipes are trademarks owned by Kerr Corporation. VITA is a trademark of VITA Zahnfabrik Company, Germany.

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